CURAFILL Anatomical Wedges

Cunei interdentali

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

CURAFILL Anatomical Wedges

Interdental wedges

Description

CURAFILL Anatomical Wedges separate teeth and prevent matrix displacement while preserving the anatomical contours of teeth and tissues of interproximal areas.

Allow good adaptation and stability of the matrix band to the irregular tooth surface.

CURAFILL Anatomical Wedges interdental wedges adapt to the anatomy of the tooth.

Method of use

1. Make sure the product is clean before use.
2. During dental restoration, select the appropriate size according to the space between the patient's teeth.
3. It is forbidden to use the product if it is damaged, deformed or if the packaging is damaged.
4. This product may only be used in a dental setting by trained personnel.

Contraindications:

At this time, no contraindications have been identified.

Precautions

- Product for dental use only.
- It should be used only by professionals who observe the procedure and directions.

Storage

- The product should be stored in a dry, clean environment at room temperature.
- Avoid direct sunlight.

Packaging

- **cod. CR490550** - Orange
- **cod. CR490551** - White
- **cod. CR490552** - Green
- **cod. CR490553** - Yellow
- **cod. CR490554** - PurpleBlu
- **cod. CR490556** - Pink

1 box CURAFILL Anatomical Wedges of 100pcs

Symbols

Product code

Lot number

Date of manufacturer

Do not reuse

Keep away from sunlight

Medical device

IFU

Manufacturer

Warnings

You must report any serious incidents occurring in relation to the medical device you supply to the manufacturer and to the competent authority of the Member State in which you are located.

Garanzia

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it