*endopost / conixpost / crownpost / shortpost R&R / shortpost R / endopost plus  
shapepost / flatpost / radiuspost / twopost / widepost / thinpost / evopost*

**Protocollo clinico**

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## Clinical procedure

1. Isolate the tooth with dental gum.
2. Partially remove gutta-percha with the usual instrument and complete the canal enlargement with a reamer.
3. Choose the size and shape of post.
4. Set the length for the post cementation.
5. Place the post with an appropriate drill.
6. Verify the post placement within the canal and establish its length.
7. Cut the post with a low speed diamond disc. Avoid the use of pliers and cutters as their action could damage the post fiber and its structure.
8. Clean the post surface with alcohol.
9. Proceed with canal and post etching, according to the manufacturer's instructions.
10. Rinse the canal and be careful all the etching material is rinsed away (we suggest to use irrigation needles).
11. Please dry the canal with paper points.
12. Apply the adhesive, the primer and the cement, according to the manufacturer's instructions. If necessary, apply a layer of cement on the post surface.
13. Place the post within the canal and gently push it down. Spread any cement excess and keep the post fixed till the cement is totally dry.
14. For core build up, please follow the manufacturer's instructions.

## Precautions

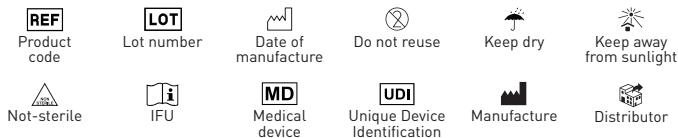
- For dental use only.
- The posts are single-use devices and cannot be reused. Mechanical and cement adhesion characteristics would not be clinically guaranteed.
- Should be used only by professionals who observe the procedure and indications, such as dental physicians and specialists in dentistry.

## Warnings

You must report any serious incidents occurring in relation to the medical device you supply to the manufacturer and to the competent authority of the Member State in which you are located.

The safety and clinical performance summary is available from EUDAMED and until this platform is fully operational can be requested from Dental World by sending an email to [info@dental-world.it](mailto:info@dental-world.it) (basic UDI-DI: 8050996PERNIENDOWD).

## Symbols

## Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection.

 Dental World S.r.l.  
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)  
Tel. +39 080 3381004 - Fax. +39 080 3386617  
e-mail: [info@dental-world.it](mailto:info@dental-world.it) / web: [www.dental-world.it](http://www.dental-world.it)

 CURADEN AG - SWITZERLAND  
Amlehnstrasse 22 - 6010 Kriens  
[www.curaden.com](http://www.curaden.com) / [info@curaden.ch](mailto:info@curaden.ch)

*endopost / conixpost / crownpost / shortpost R&R / shortpost R / endopost plus  
shapepost / flatpost / radiuspost / twopost / widepost / thinpost / evopost*

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