# IT - Cura Carrier

## Strumenti canalari meccanici

### Descrizione

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# EN - Cura Carrier

## Mechanical root canal instruments

### Description

Cura Carrier with spring is the ideal lentulo, made of stainless steel, for introducing cements and medicaments into canals. Flexibility and safety are guaranteed thanks to the short spring between the spiral and the shank.

### Indications

For use in dental practices or hospitals by qualified personnel only.

### Field of application

For catheterisation and shape definition.

### Measurements

Cura Carrier			
	21 mm	25 mm	29 mm
● 25	155550915	155550925	155550935
● 30	155513505	155512100	155514010
● 35	155513515	155512110	155512020
● 40	155513525	155512120	155514030
06-15	155513535	155512130	155514040

### Packaging

Non-sterile device in blister pack of 4 pieces (assorted or individual sizes).

### Recommended procedure

#### Operating protocol:

- Insert the cement-covered instrument into the canal and then slowly pull it out in a rotary motion.
- During the endodontic sequence, irrigate repeatedly with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal sludge.

### Storage

Store at room temperature, away from moisture and direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials. It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

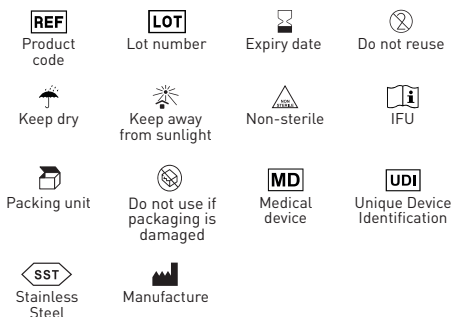
### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.  
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)  
Tel. +39 080 3381004 - Fax. +39 080 3386617  
e-mail: [info@dental-world.it](mailto:info@dental-world.it) / web: [www.dental-world.it](http://www.dental-world.it)

 Curaden AG - Switzerland  
Amlehnstrasse 22 - 6010 Kriens  
[www.curaden.com](http://www.curaden.com) / [info@curaden.ch](mailto:info@curaden.ch)

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