# IT - Cura Flayre

## Strumenti canalari meccanici

### Descrizione

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# EN - Cura Flayre

## Mechanical root canal instruments

### Description

Cura Flayre is a Ni-Ti instrument indicated for opening the coronal portion of the canal. This procedure is indispensable to avoid tension on the instruments and to limit any risk of premature instrument wear.

### Indications

To be used only in a dental practice or hospital environment by qualified personnel.

### Field of application

Shaping and cleaning root canals.

### Measures

Cura Flayre					
	∅	%	N/cm	RPM	15 mm
25	25	12	1,5/3	300 - 600	<b>155900420</b>

### Packaging

Non-sterile device in blister pack of 6 pieces

### Recommended procedure

#### Operating protocol:

- Locate the canal.
- Use the Cura Flayre instrument to widen the mouth of the root canal, thus eliminating dentinal protrusions and preserving the homogeneity of the root canal. This step is essential to avoid subsequent stress on the shaping instruments and thus limit the risk of premature instrument failure. Coronal countersinking simplifies changes in working lengths and facilitates the insertion of instruments through the apical third and thus facilitates access to subsequent instrumentation and canal irrigation.
- Then follow the operative sequence most suitable for the case to be treated.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. to remove all necrotic tissue and the organic component of the dentinal sludge.

### Storage

Store at room temperature, away from moisture and direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials. It is recommended to always use surgical gloves for personal protection against bacterial

contamination. Failure to do so may result in cross-infection.

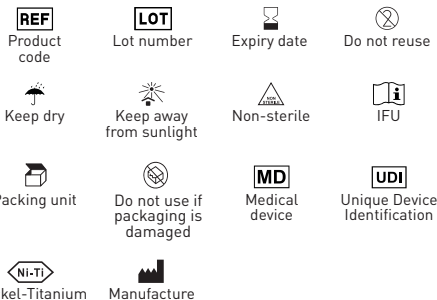
### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.  
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)  
Tel. +39 080 3381004 - Fax. +39 080 3386617  
e-mail: info@dental-world.it / web: www.dental-world.it

 Curaden AG - Switzerland  
Amlehnstrasse 22 - 6010 Kriens  
www.curaden.com / info@curaden.ch

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