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Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Curaden AG - Switzerland

Amlehnstrasse 22 - 6010 Kriens

www.curaden.com / info@curaden.ch

EN - Cura Flex N.H.A.

Mechanical root canal instruments

Description

Cura Flex N.H.A. root canal instruments are thermally activated Ni-Ti and Coated files designed for complete root canal preparation. The kit includes an opening instrument, the Cura Pilot N.H.A. glide-path instrument and four rotary files for shaping and finishing all types of canals.

Indications

For use only in a dental practice or hospital environment by qualified personnel.

Field of application

Glide path, shaping and cleaning root canals.

Measures

Cura Flex NHA								
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm	
R	20	10	2,5	350	17 mm	REF: 155900550		
D1	20	4	2,5	150-350	155900552	155900562	155900572	
D2	25	4	2,5	150-350	155900553	155900563	155900573	
D3	30	4	2,5	150-350	155900554	155900564	155900574	
D4	25	6	2,5	150-350	155900555	155900565	155900575	
R-D4	--	--	2,5	--	155900556	155900566	155900576	
D5	35	4	2,5	150-350	155900557	155900567	155900577	
D6	40	4	2,5	150-350	155900558	155900568	155900578	
D7	50	4	2,5	150-350	155900559	155900569	155900579	
D5-D7	--	--	2,5	--	155900560	155900570	155900580	

Packaging

Non-sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Locate the canal.
- Gently perform a passive probing of the coronal 2/3 with a hand instrument ISO 010 or 015 (mechanical files such as Cura Pat can be used alternatively).
- Using the R instrument, create access to the canal orifice and irrigate.
- Use sodium hypochlorite and viscous chelator as a lubricant to irrigate the canal.
- To achieve a reproducible glide-path before enlargement and to reduce the workload of the shaping instruments and the risk of breakage, work gently with the Cura Pilot N.H.A. rotary instrument (ISO 016).
- Shape with D1 until working length is reached, irrigate.
- Use D2 to reconfirm the working length, recapitulate and irrigate again, especially in more curved canals.
- For large canals use D3/D4 without oscillating movement.
- For finishing rather large root canals use instruments D5, D6 and D7.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

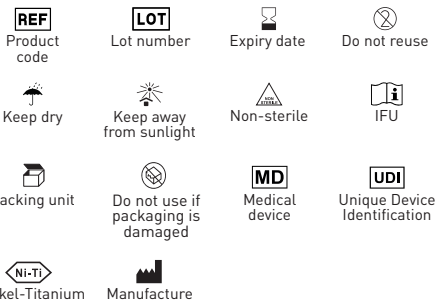
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

Curaden AG - Switzerland
Amlehnstrasse 22 - 6010 Kriens
www.curaden.com / info@curaden.ch

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