IT - Cura Flexo File

Strumenti canalari manuali

Descrizione

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Manual root canal instruments

Description

Cura Flexo File are super-flexible steel files indicated for the preparation of irregular canals.

Indications

Only to be used in a dental practice or hospital environment by qualified personnel.

Field of application

For catheterisation and shape definition.

Measures

Cura Flexo File		
	21mm	25mm
○ 15	155517182	155517242
● 20	155517192	155517252
● 25	155517202	155517262
● 30	155517212	155517272
● 35	155517222	155517282
● 40	155517232	155517292
15-40	155550712	155550732

Packaging

Non-sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

Use the Cura Flexo File for opening and shaping the root canal. Made of stainless steel, they provide greater flexibility for root canal cleaning. The triangular section facilitates the removal of residues. Work in quarter rotation, then withdraw the instrument.

During the endodontic sequence, irrigate repeatedly with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal sludge.

Storage

Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination.

Failure to do so may result in cross-infection.

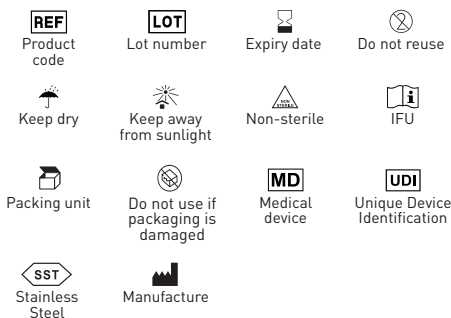
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

Curaden AG - Switzerland
Amlehnstrasse 22 - 6010 Kriens
www.curaden.com / info@curaden.ch

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