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Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Curaden AG - Switzerland

Amlehnstrasse 22 - 6010 Kriens

www.curaden.com / info@curaden.ch

EN - Cura Pilot N.H.A. - Cura Pilot H.A.

Mechanical root canal instruments

Description

Cura Pilot H.A. and Cura Pilot N.H.A. are Ni-Ti instruments available in Gold, Blue and Coated, with thermal activation. They offer excellent flexibility by easily adapting to the anatomy of most root canals. Instruments with progressively variable taper allow you to work with greater confidence in establishing the glide path.

Indications

To be used only in a dental practice or hospital environment by qualified personnel.

Field of application

Glide path creation.

Measures

Cura Pilot NHA							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
○ 016	15	2-6	2,5	300	155900551	155900561	155900571

Cura Pilot Heat Activation							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
○ 016	15	2-6	2,5	300	155900751	155900761	155900771

Cura Pilot Heat Activation Gold							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
○ 016	15	2-6	2,5	300	155900660	155900670	155900680

Packaging

Non-sterile device in blister pack of 6 pieces.

Recommended procedure

Operating protocol:

- Locate the canal.
- Use a coronal portion opening tool (Cura Flayre).
- Probe with an ISO 010 or 015 manual file.
- Work gently with the Cura Pilot H.A. or Cura Pilot N.H.A. rotary instrument (ISO 016) in order to obtain a reproducible glide path before canal widening and to reduce the workload of the shaping instruments and risk of breakage.
- Flush the canal using sodium hypochlorite and viscous chelator as lubricant.
- Then follow the operative sequence most suitable for the case to be treated.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. to remove all necrotic tissue and the organic component of the dentinal sludge.

Storage

Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for

personal protection against bacterial contamination. Failure to do so may result in cross-infection.

Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use with patients with known allergic sensitivity to raw materials.

Symbols

Product code

Lot number

Expiry date

Do not reuse

Keep dry

Keep away from sunlight

Non-sterile

IFU

Packing unit

Do not use if packaging is damaged

Medical device

Unique Device Identification

Nickel-Titanium

Manufacture

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

Curaden AG - Switzerland
Amlehstrasse 22 - 6010 Kriens
www.curaden.com / info@curaden.ch

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