IT - Cura Shaper Gold

Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Motfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Curaden AG - Switzerland

Amlehnstrasse 22 - 6010 Kriens

www.curaden.com / info@curaden.ch

EN - Cura Shaper Gold

Mechanical root canal instruments

Description

Cura Shaper Gold root canal instruments are made with Ni-Ti Evo Brown technology, which gives greater elasticity and superior strength compared to normal Ni-Ti instruments. This special alloy This special alloy guarantees extraordinary resistance to cyclic fatigue due to the innovative heat treatment process, as well as greater flexibility, heat treatment process, as well as greater flexibility, ensuring that the canal morphology is maintained.

Indications

For use in dental practices or hospitals by qualified personnel only.

Field of application

Shaping and cleaning of root canals.

Measures

Cura Shaper Gold							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
DX	19	3,5-19	3/4	150-300	19 mm	REF: 155900340	
● D1	17	0,2-11	3/4	150-300	155900341	155900351	155900361
○ D2	20	0,4-11,5	1/1,5	150-300	155900342	155900352	155900362
● M1	20	0,7	1/1,5	150-300	155900343	155900353	155900363
● M2	25	0,8	2,5	150-300	155900344	155900354	155900364
● M3	30	0,9	2,5	150-300	155900345	155900355	155900365
●● M4	40	0,6	2,5	150-300	155900346	155900356	155900366
●● M5	50	0,5	2,5	150-300	155900347	155900357	155900367
DX-M3	--	--	--	150-300	155900348	155900358	155900368

Packaging

Non-sterile device in blister pack of 6 pieces (assorted or individual sizes)

Recommended procedure

Operating protocol:

- Locate the canal.
- Use the DX opening tool to widen the canal.
- Probe the canal with an ISO 010 or ISO 015 hand file until resistance is encountered resistance.
- Flush the canal using sodium hypochlorite and viscous chelator as a lubricant.
- For shaping, use the D1 instrument until the working length determined previously determined.
- Use shaping tool D2 to reconfirm the working length.
- Check the working length again by probing with an ISO 010/015 tool.
- Use the finishing tool M1 until the working length is reached.
- Measure the channel using a manual file.
- If further widening of the apical foramen is necessary, use the appropriate finishing instrument appropriate finishing instrument (M2, M3) according to the working length.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

Store at room temperature, away from moisture and

direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

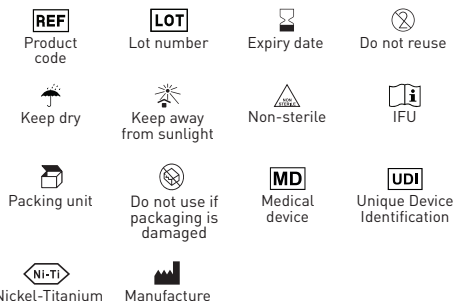
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

Curaden AG - Switzerland
Amlehstrasse 22 - 6010 Kriens
www.curaden.com / info@curaden.ch

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