# IT - Cura Spreader SS e Ni-Ti

## Strumenti canalari manuali

### Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Curaden AG - Switzerland  
Amlehnstrasse 22 - 6010 Kriens  
www.curaden.com / info@curaden.ch

# EN - Cura Spreader SS e Ni-Ti

## Manual root canal instruments

### Description

Spreader are steel and nickel titanium instruments with a conical shape and a sharp tip.  
Suitable for lateral condensation of gutta-percha.

### Indications

For use in dental practices or hospitals by qualified personnel only.

### Field of application

For lateral condensation of gutta-percha.

### Measures

SPREADER - Stainless Steel		
	21mm	25mm
○ 15	155513142	155513242
● 20	155513112	155513212
● 25	155513152	155513252
● 30	155513122	155513222
● 35	155513162	155513262
● 30	155513132	155513232
15-40	155550832	155550842

SPREADER - Nickel-Titanium	
	25mm
○ 15	155443242
● 20	155443212
● 25	155443252
● 30	155443222
● 35	155443262
● 30	155443232
15-40	155450842

### Packaging

Non-sterile device in blister packs of 6 pieces (assorted or individual sizes).

### Recommended procedure

#### Operating protocol:

Insert a master cone of appropriate size and taper until resistance is felt to insertion. The master taper is chosen in relation to the instrument used to shape the canal. Check radiographically for correct positioning and remove the cone. Mix the cement canal cement and insert it into the canal, then introduce the master cone into the canal again. Place the master cone, which is laterally compacted using the spreader.

During the endodontic sequence, irrigate repeatedly with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal sludge.

### Storage

Store at room temperature, away from moisture and direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination.

Failure to do so may result in cross-infection.

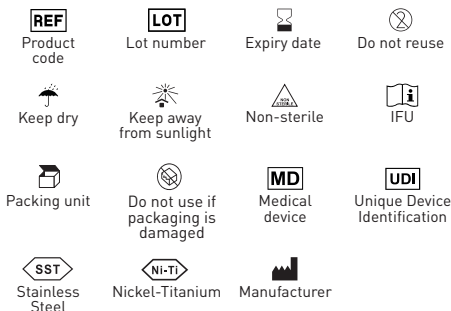
### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device you have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for careless use.

Dental World S.r.l.  
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)  
Tel. +39 080 3381004 - Fax. +39 080 3386617  
e-mail: info@dental-world.it / web: www.dental-world.it

Curaden AG - Switzerland  
Amlehnstrasse 22 - 6010 Kriens  
www.curaden.com / info@curaden.ch

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