# IT - Cura Carrier

## Strumenti canalari meccanici

### Descrizione

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# EN - Cura Carrier

## Mechanical root canal instruments

### Description

Cura Carrier with spring is the ideal lentulo, made of stainless steel, for introducing cements and medicaments into canals. Flexibility and safety are guaranteed thanks to the short spring between the spiral and the shank.

### Indications

For use in dental practices or hospitals by qualified personnel only.

### Field of application

For catheterisation and shape definition.

### Measurements

Cura CARRIER			
	21 mm	25 mm	29 mm
● 25	155513505S	155512100S	155514010S
● 30	155513515S	155512110S	155514020S
● 35	155513525S	155512120S	155514030S
● 40	155513535S	155512130S	155514040S
06-15	155550915S	155550925S	155550935S

### Packaging

Sterile device in blister pack of 4 pieces (assorted or individual sizes).

### Recommended procedure

#### Operating protocol:

- Insert the cement-covered instrument into the canal and then slowly pull it out in a rotary motion.
- During the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. to remove all necrotic tissue and the organic component of the dentinal slurry.

### Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials. It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

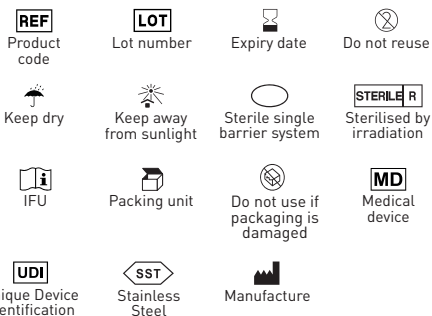
### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.  
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)  
Tel. +39 080 3381004 - Fax. +39 080 3386617  
e-mail: info@dental-world.it / web: www.dental-world.it

 Curaden AG - Switzerland  
Amlehnstrasse 22 - 6010 Kriens  
www.curaden.com / info@curaden.ch

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