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Strumenti canalari meccanici

Descrizione

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EN - Cura Flex N.H.A.

Mechanical root canal instruments

Description

Cura Flex N.H.A. root canal instruments are thermally activated Ni-Ti and Coated files designed for complete root canal preparation. The kit includes an opening instrument, the Cura Pilot N.H.A. glide-path instrument and four rotary files for shaping and finishing all types of canals.

Indications

For use only in a dental practice or hospital environment by qualified personnel.

Field of application

Glide path, shaping and cleaning root canals.

Measures

Cura Flex NHA							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
R	20	10	2,5	350	17 mm	REF. 155900550S	
● D1	20	4	2,5	150-350	155900552S	155900562S	155900572S
● D2	25	4	2,5	150-350	155900553S	155900563S	155900573S
● D3	30	4	2,5	150-350	155900554S	155900564S	155900574S
● D4	25	6	2,5	150-350	155900555S	155900565S	155900575S
R-D4	--	--	2,5	--	155900556S	155900566S	155900576S
● D5	35	4	2,5	150-350	155900557S	155900567S	155900577S
● D6	40	4	2,5	150-350	155900558S	155900568S	155900578S
● D7	50	4	2,5	150-350	155900559S	155900569S	155900579S
D5-D7	--	--	2,5	--	155900560S	155900570S	155900580S

Packaging

Sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Locate the canal.
- Gently perform a passive probing of the coronal 2/3 with a hand instrument ISO 010 or 015 (mechanical files such as Cura Pat can be used alternatively).
- Using the R instrument, create access to the canal orifice and irrigate.
- Use sodium hypochlorite and viscous chelator as a lubricant to irrigate the canal.
- To achieve a reproducible glide-path before enlargement and to reduce the workload of the shaping instruments and the risk of breakage, work gently with the Cura Pilot N.H.A. rotary instrument (ISO 016).
- Shape with D1 until working length is reached, irrigate.
- Use D2 to reconfirm the working length, recapitulate and irrigate again, especially in more curved canals.
- For large canals use D3/D4 without oscillating movement.
- For finishing rather large root canals use instruments D5, D6 and D7.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

The blister should only be opened at the time of surgery under controlled aseptic conditions. Store at room

temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

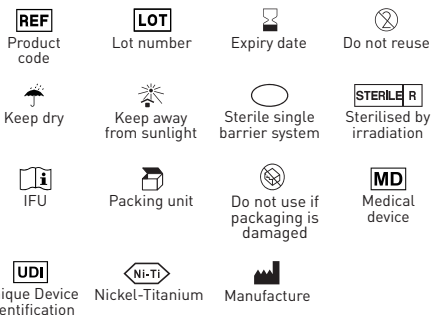
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

Curaden AG - Switzerland
Amlehstrasse 22 - 6010 Kriens
www.curaden.com / info@curaden.ch

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