IT - Cura Race H.A.

Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Curaden AG - Switzerland

Amlehstrasse 22 - 6010 Kriens

www.curaden.com / info@curaden.ch

EN - Cura Race H.A.

Mechanical root canal instruments

Description

Cura Race H.A. root canal instruments are made with Ni-Ti C New-Wire technology, which provides greater elasticity and superior strength compared to normal Ni-Ti instruments.

Indications

For use in dental practices or hospitals by qualified personnel only.

Field of application

Shaping and cleaning root canals.

Measures

Cura Race Heat Activation					
	Ø	%	21 mm	25 mm	31 mm
● R25	25	0,8	155900710S	155900720S	155900725S
● R40	40	0,6	155900711S	155900721S	155900726S
● R50	50	0,5	155900712S	155900722S	155900727S
R25-R50	--	--	155900713S	155900723S	155900728S

Packaging

Sterile device in blister pack of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Prepare a root canal access.
- Select the Cura Race H.A. instrument to be used for canal preparation.
- If the canal is considered narrow use the R25 instrument. If the canal is considered medium use the R40 instrument. If the canal is considered wide use the R50 instrument.
- Preparation with the R25 instrument: estimate the length of the root canal with the help of the radiograph and adjust the stop on the instrument to approx. 2/3 of the estimated length.
- Preparation with the R40-R50 instrument: electronically determine the working length and adjust the stop on the instrument to this length.
- Irrigate the canal with E.D.T.A. and hypochlorite.
- Insert the instrument into the canal and move it slowly apically, applying very light pressure; the instrument can then move forward easily. The amplitude of the up and down movements (passages) must not exceed 3 mm.
- After 3 passes clean the instrument, irrigate the canal and check its patency.
- Repeat until 2/3 of the estimated working length is reached.
- If using the R25 instrument, determine the working length electronically. If you are using the R40 and R50, after preparing 2/3 of the channel length, electronically check the working length initially established.
- Subsequently, recapitulate until the entire working length is reached.
- Determine the apical diameter by inserting a hand instrument ISO higher than the Cura Race H.A. instrument inserted up to the working length. If the hand instrument is inserted up to 0.7 mm before the working length, no further extension of the apical area is necessary. If the hand instrument can be inserted up to the working length, it is usually necessary to finish the preparation with a larger Cura Race H.A. instrument.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

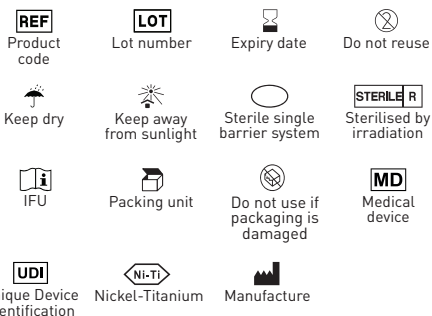
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

 Curaden AG - Switzerland
Amlehnstrasse 22 - 6010 Kriens
www.curaden.com / info@curaden.ch

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