IT - Cura Shaper 2 Gold

Strumenti canalari meccanici

Descrizione

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EN - Cura Shaper 2 Gold

Mechanical root canal instruments

Description

Cura Shaper 2 Gold root canal instruments are made with Ni-Ti Evo Brown technology, which gives greater elasticity and strength and follows even the most pronounced curves harmoniously, compared with normal Ni-Ti instruments. This new technology shortens the time spent changing instruments; in addition, continuous clockwise mechanical instrument movement is recommended.

Indications

For use in dental practices or hospitals by qualified personnel only.

Field of application

Shaping and cleaning of root canals.

Measures

Cura Shaper 2 Gold							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
● X1	17	0,4	2;5,2	300	155900300S	155900310S	155900320S
● X2	25	0,5	2;5,2	300	155900301S	155900311S	155900321S
● X3	35	0,7	2;5,2	300	155900302S	155900312S	155900322S
●● X4	40	0,6	2;5,2	300	155900303S	155900313S	155900323S
●● X5	50	0,6	2;5,2	300	155900304S	155900314S	155900324S
Assorted	--	--	--	--	155900305S	155900315S	155900325S

Packaging

Sterile device in blister pack of 6 pieces (assorted or individual sizes)

Recommended procedure

Operating protocol:

- Locate the canal.
- Use an opening tool to widen the canal or mechanically driven glide-path files (Cura Pat).
- Probe the canal with an ISO 010 or ISO 015 hand file until resistance is met.
- Flush the canal using sodium hypochlorite and viscous chelator as lubricant.
- For shaping, use the X1 shaping tool until the working length determined beforehand is reached.
- Use shaping tool X2 to reconfirm the working length.
- Check the working length again by probing with an ISO 010/015 tool.
- Use the finishing tool X3 until the working length is reached.
- Measure the channel with a manual file.
- If further widening of the apical foramen is necessary, use the appropriate finishing instrument (X4, X5) according to the working length.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Product code

Lot number

Expiry date

Do not reuse

Keep dry

Keep away from sunlight

Sterile single barrier system

Sterilised by irradiation

IFU

Packing unit

Do not use if packaging is damaged

Medical device

Unique Device Identification

Nickel-Titanium

Manufacture

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

Curaden AG - Switzerland
Amlehnstrasse 22 - 6010 Kriens
www.curaden.com / info@curaden.ch

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