# IT - Cura Shaper H.A.

## Strumenti canalari meccanici

### Descrizione

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# EN - Cura Shaper H.A.

## Mechanical root canal instruments

### Description

Cura Shaper H.A. root canal instruments are made with Ni-Ti C New-Wire technology, which gives greater elasticity and superior strength compared to normal Ni-Ti instruments. This special alloy This special alloy guarantees extraordinary resistance to cyclic fatigue due to the innovative heat treatment process, as well as greater flexibility. heat treatment process, as well as greater flexibility, ensuring that the canal morphology is maintained.

### Indications

For use in dental practices or hospitals by qualified personnel only.

### Field of application

Shaping and cleaning of root canals.

### Measures

Cura Shaper Heat Activation									
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm		
DX	19	3,5-19	2,0-2,5	150-300	19 mm		REF: 1559006005		
D1	17	0,2-11	2,0-2,5	150-300	1559006015	1559006105	1559006205		
D2	20	0,4-11,5	2,0-2,5	150-300	1559006025	1559006115	1559006215		
M1	20	0,7	2,0-2,5	150-300	1559006035	1559006125	1559006225		
M2	25	0,8	2,0-2,5	150-300	1559006045	1559006135	1559006235		
M3	30	0,9	2,0-2,5	150-300	1559006055	1559006145	1559006245		
DX-M3	--	--	--	150-300	1559006065	1559006155	1559006255		

### Packaging

Sterile device in blister pack of 6 pieces (assorted or individual sizes).

### Recommended procedure

#### Operating protocol:

- Locate the canal.
- Use the DX opening tool to widen the canal.
- Probe the canal with an ISO 010 or ISO 015 hand file until resistance is encountered resistance.
- Flush the canal using sodium hypochlorite and viscous chelator as a lubricant.
- For shaping, use the D1 instrument until the working length determined previously determined.
- Use shaping tool D2 to reconfirm the working length.
- Check the working length again by probing with an ISO 010/015 tool.
- Use the finishing tool M1 until the working length is reached.
- Measure the channel using a manual file.
- If further widening of the apical foramen is necessary, use the appropriate finishing instrument appropriate finishing instrument (M2, M3) according to the working length.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal slurry.

### Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

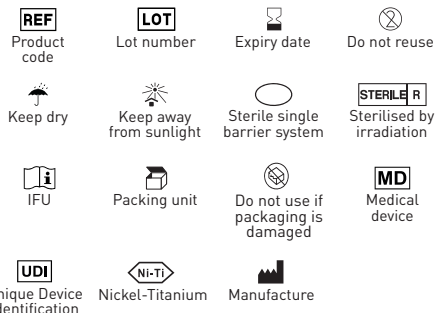
### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.  
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)  
Tel. +39 080 3381004 - Fax. +39 080 3386617  
e-mail: info@dental-world.it / web: www.dental-world.it

Curaden AG - Switzerland  
Amlehnstrasse 22 - 6010 Kriens  
www.curaden.com / info@curaden.ch

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