ELASTIC DENTAL dam

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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INSTRUCTIONS FOR USE DENTAL DAM - LATEX - POWDER-FREE

Description
Elastic Dental Dam is a medical device used by dentists to perform therapeutic procedures. It comes in the form of thin, colored latex sheets that are placed at the beginning of the dental session to insulate the teeth during treatment.

- Recommended procedure**
- Determine the affected tooth and drill holes of about 2mm on the dam sheet, through which to pass the teeth on which to perform the treatment.
 - Use pliers to hold/widen the clamps and place the metal clamps in the hole.
 - Place the dental dam with clamps on the affected tooth.
 - Place the frame of the dental dam outside the mouth to retain excess material from the dam.

- Storage**
- Do not store in high temperature or bright light.
 - Store in a cool place.
 - Store preferably at temperatures below 40°C.

- Precautions and Warnings**
- Disposable. Do not reuse the device to avoid causing cross-contamination or infection.
 - Product for dental use only.
 - Must be used only by professionals who observe the procedure and directions.
 - Keep away from non-professional users.
 - This product contains natural rubber latex that may cause allergic reactions. Improper use may cause allergic reactions or skin and tissue irritation.
 - Do not use on patients who have demonstrated hypersensitivity and allergic reactions prior to use.
 - The product may not be used for purposes other than that for which it was designed.
 - Before using the product, medical personnel should check the integrity of the packaging.
 - Disposal of used, deteriorated or expired product must be done according to regulations about dental waste disposal.
 - Do not use the product after the expiration date.
 - You must report any serious incidents that have occurred in connection with the medical device we supplied to the manufacturer and to the competent authority of the member state where you are located.

- Packaging (1 box = 36 sheets):**
- Elastic Dental Dam - Green - Thin REF 00340800
 - Elastic Dental Dam - Green - Medium REF 00340801
 - Elastic Dental Dam - Green - Heavy REF 00340802
 - Elastic Dental Dam - Blue - Medium REF 00340803
 - Elastic Dental Dam - Blue Eyes - Medium REF 00340804
 - Elastic Dental Dam - Black - Medium REF 00340808

Symbols

Product code	Lot number	Expiry date	Do not reuse	Contains latex	Caution
Keep dry	Keep away from sunlight	Upper limit of temperature	Non-sterile	IFU	Medical device
Unique Device Identification	Manufacturer				

Warranty
Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

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Precautions and Warnings

- Disposable. Do not reuse the device to avoid causing cross-contamination or infection.
- Product for dental use only.
- Must be used only by professionals who observe the procedure and directions.
- Keep away from non-professional users.
- This product contains natural rubber latex that may cause allergic reactions. Improper use may cause allergic reactions or skin and tissue irritation.
- Do not use on patients who have demonstrated hypersensitivity and allergic reactions prior to use.
- The product may not be used for purposes other than that for which it was designed.
- Before using the product, medical personnel should check the integrity of the packaging.
- Disposal of used, deteriorated or expired product must be done according to regulations about dental waste disposal.
- Do not use the product after the expiration date.
- You must report any serious incidents that have occurred in connection with the medical device you supplied to the manufacturer and to the competent authority of the member state where you are located.

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