IT - ELI Flex Blu

Strumenti canalari meccanici

Descrizione

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EN - ELI Flex Blu

Mechanical root canal instruments

Description

ELI Flex Blu root canal instruments comprise just three instruments that can be used in succession to shape and finish all types of canals.

Indications

For use only in a dental practice or hospital environment by qualified personnel.

Field of application

Shaping and cleaning of root canals.

Measures

ELI Flex Blu							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
R	--	--	2,5	350	17 mm		REF: 7300371
S1	20	0,4	2,5	250-300	7300372	7300376	7300380
S2	25	0,6	2,5	250-300	7300373	7300377	7300381
S3	35	0,4	2,5	250-300	7300374	7300378	7300382
10-S3	--	--	2,5	--	7300375	7300379	7300383

Packaging

Non-sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Locate the canal.
- Using the R instrument, create access to the canal orifice and irrigate.
- Gently perform a passive probing of the coronal 2/3 with a hand instrument ISO 010 or 015 (mechanical files such as ELI Pat can be used alternatively).
- Use sodium hypochlorite and viscous chelator as a lubricant to irrigate the canal.
- Work carefully and gently until a reproducible glide-path is obtained.
- Shape with S1 until working length is reached, irrigate.
- Use S2 to reconfirm the working length, recapitulate and irrigate again, especially in more curved canals.
- For large canals use S3 without oscillating movement.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials. It is recommended to always use surgical

gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

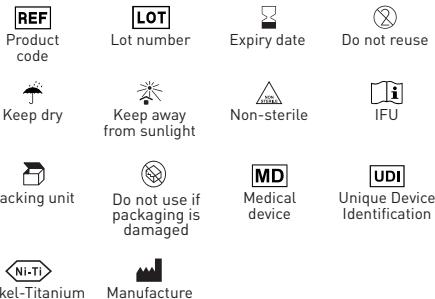
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

Eli-Dent Group S.p.A.
Via Guido Rossa, 24 A/B/C - 50032
Borgo San Lorenzo (FI) - www.elidentgroup.it
Tel. +39 055 600545 - Fax +39 055 601069

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