# IT - ELI One

## Strumenti canalari meccanici

### Descrizione

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Tel. +39 055 600545 - Fax +39 055 601069

# EN - ELI One

## Mechanical root canal instruments

### Description

ELI One root canal instruments are made with New-Wire Ni-Ti technology, which provides high elasticity and superior strength compared to normal Ni-Ti instruments.

### Indications

To be used only in a dental practice or hospital environment by qualified personnel.

### Field of application

Shaping and cleaning root canals.

### Measures

ELI One					
	∅	%	21 mm	25 mm	31 mm
● 21	21	0,6	<b>7300985</b>	<b>7300989</b>	<b>7300993</b>
● 25	25	0,8	<b>7300986</b>	<b>7300990</b>	<b>7300994</b>
● 40	40	0,8	<b>7300987</b>	<b>7300991</b>	<b>7300995</b>
21-40	--	--	<b>7300988</b>	<b>7300992</b>	<b>7300996</b>

### Packaging

Non-sterile device in blister pack of 3 pieces (assorted or individual sizes).

### Recommended procedure

#### Operating protocol:

- Locate the canal.
- Create access to the canal using an opening tool (ELI Flayre).
- Probe the channel using a manual ISO 010 instrument and expand it to 015, with manual files until resistance is met to determine the working length.
- Start treatment using the 25/08 shaping instrument, irrigate the canal using sodium hypochlorite and viscous chelating agent as lubricant.
- Shape to approx. 2-3 mm, pull out the instrument and repeat the operation in each canal, irrigate again, recapitulate.
- Check the working length using hand instrument 010 with the help of E.D.T.A.
- Bring instrument 25/08 to working length again without forcing. If the tool does not advance use the 21/06 for 2/3 passes until there is no more debris.
- If, however, there is no more debris when using the 25/08 instrument, shape it with the 40/08.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal slurry.

### Storage

Store at room temperature, away from moisture and

direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

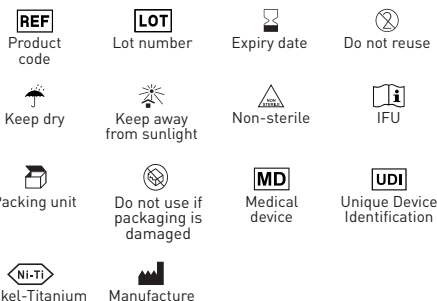
### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.  
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)  
Tel. +39 080 3381004 - Fax. +39 080 3386617  
e-mail: info@dental-world.it / web: www.dental-world.it

 Eli-Dent Group S.p.A.  
Via Guido Rossa, 24 A/B/C - 50032  
Borgo San Lorenzo (FI) - www.elidentgroup.it  
Tel. +39 055 600545 - Fax +39 055 601069

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