IT - ELI Pilot Evolution - ELI Pilot Active

Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Eli-Dent Group S.p.A.

Via Guido Rossa, 24 A/B/C - 50032

Borgo San Lorenzo (FI) - www.elidentgroup.it

Tel. +39 055 600545 - Fax +39 055 601069

EN - ELI Pilot Evolution - ELI Pilot Active

Mechanical root canal instruments

Description

ELI Pilot Active and ELI Pilot Evolution are Ni-Ti instruments available in Gold, Blue and Coated, with thermal activation. They offer excellent flexibility by easily adapting to the anatomy of most root canals. Instruments with progressively variable taper allow you to work with greater confidence in establishing the glide path.

Indications

To be used only in a dental practice or hospital environment by qualified personnel.

Field of application

Glide path creation.

Measures

ELI Pilot Evolution							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
○ 016	15	2-6	2,5	300	7301038	7301048	7301058

ELI Pilot Active Blu							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
○ 016	15	2-6	2,5	300	7301105	7301106	7301107

ELI Pilot Active Gold							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
○ 016	15	2-6	2,5	300	7301108	7301109	7301110

Packaging

Non-sterile device in blister pack of 6 pieces.

Recommended procedure

Operating protocol:

- Locate the canal.
- Use a coronal portion opening tool (ELI Flayre).
- Probe with an ISO 010 or 015 manual file.
- Work gently with the ELI Pilot Active or ELI Pilot Evolution rotary instrument (ISO 016) in order to obtain a reproducible glide path before canal widening and to reduce the workload of the shaping instruments and risk of breakage.
- Flush the canal using sodium hypochlorite and viscous chelator as lubricant.
- Then follow the operative sequence most suitable for the case to be treated.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. to remove all necrotic tissue and the organic component of the dental sludge.

Storage

Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials. It is recommended to always use surgical gloves for personal protection against bacterial contamination.

Failure to do so may result in cross-infection.

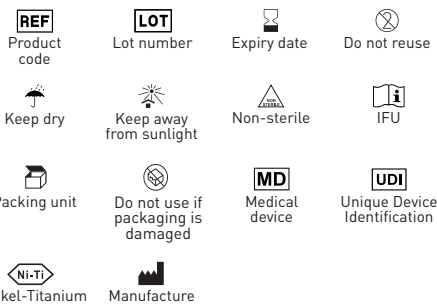
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

Eli-Dent Group S.p.A.
Via Guido Rossa, 24 A/B/C - 50032
Borgo San Lorenzo (FI) - www.elidentgroup.it
Tel. +39 055 600545 - Fax +39 055 601069

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