IT - ELI Proc

Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Eli-Dent Group S.p.A.

Via Guido Rossa, 24 A/B/C - 50032

Borgo San Lorenzo (FI) - www.elidentgroup.it

Tel. +39 055 600545 - Fax +39 055 601069

EN - ELI Proc

Mechanical root canal instruments

Description

ELI Proc root canal instruments are made of Ni-Ti New-Wire technology, which gives extreme flexibility and strength. Thanks to the reciprocating action, only one instrument is needed for a complete root canal treatment.

Indications

Only to be used in a dental practice or hospital environment by qualified personnel.

Field of application

Shaping and cleaning of root canals.

Measures

ELI Proc					
	∅	%	21 mm	25 mm	31 mm
● R25	25	0,8	7300090	7300190	7300290
● R40	40	0,6	7300091	7300191	7300291
● R50	50	0,5	7300092	7300192	7300292
R25-R50	--	--	7300093	7300193	7300293

Packaging

Non-sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Prepare a root canal access.
- Select the ELI Proc instrument to be used for canal preparation.
- If the canal is considered narrow use the R25 instrument. If the canal is considered medium use the R40 instrument. If the canal is considered wide use the R50 instrument.
- Preparation with the R25 instrument: estimate the length of the root canal with the help of the radiograph and adjust the stop on the instrument to approx. 2/3 of the estimated length.
- Preparation with the R40-R50 instrument: electronically determine the working length and adjust the stop on the instrument to this length.
- Irrigate the canal with E.D.T.A. and hypochlorite.
- Insert the instrument into the canal and move it slowly apically, applying very light pressure; the instrument can then move forward easily. The amplitude of the up and down movements (passages) must not exceed 3 mm.
- After 3 passes clean the instrument, irrigate the canal and check its patency.
- Repeat until 2/3 of the estimated working length is reached.
- If using the R25 instrument, determine the working length electronically. If you are using the R40 and R50 instrument, after preparing 2/3 of the working length. If using the R40 and R50 instrument, after preparing 2/3 of the channel length, electronically check the working length initially established.
- Subsequently, recapitulate until the entire working length is reached.
- Determine the apical diameter by inserting a hand instrument ISO higher than the ELI Proc instrument inserted up to the working length. If the hand instrument is inserted up to 0.7 mm before the working length, no further extension of the apical area is required. If the hand instrument can be inserted up to the working length, it is usually necessary to finish the preparation

with a larger ELI Proc instrument.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

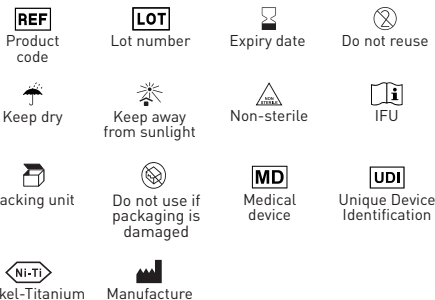
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

Eli-Dent Group S.p.A.
Via Guido Rossa, 24 A/B/C - 50032
Borgo San Lorenzo (FI) - www.elidentgroup.it
Tel. +39 055 600545 - Fax +39 055 601069

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