# IT - ELI Taper 2 Gold

## Strumenti canalari meccanici

### Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Eli-Dent Group S.p.A.

Via Guido Rossa, 24 A/B/C - 50032

Borgo San Lorenzo (FI) - www.elidentgroup.it

Tel. +39 055 600545 - Fax +39 055 601069

# EN - ELI Taper 2 Gold

## Mechanical root canal instruments

### Description

ELI Taper 2 Gold root canal instruments are made with Ni-Ti Evo Brown technology, which gives greater elasticity and strength and follows even the most pronounced curves harmoniously, compared with normal Ni-Ti instruments. This new technology shortens the time spent changing instruments; in addition, continuous clockwise mechanical instrument movement is recommended.

### Indications

For use in dental practices or hospitals by qualified personnel only.

### Field of application

Shaping and cleaning of root canals.

### Measures

ELI Taper 2 Gold							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
● X1	17	0,4	2;5,2	300	7300960	7300966	7300972
● X2	25	0,5	2;5,2	300	7300961	7300967	7300973
● X3	35	0,7	2;5,2	300	7300962	7300968	7300974
● X4	40	0,6	2;5,2	300	7300963	7300969	7300975
● X5	50	0,6	2;5,2	300	7300964	7300970	7300976
Assorted	--	--	--	--	7300965	7300971	7300977

### Packaging

Non-sterile device in blister pack of 6 pieces (assorted or individual sizes)

### Recommended procedure

#### Operating protocol:

- Locate the canal.
- Use an opening tool to widen the canal or mechanically driven glide-path files (ELI Pat).
- Probe the canal with an ISO 010 or ISO 015 hand file until resistance is met.
- Flush the canal using sodium hypochlorite and viscous chelator as lubricant.
- For shaping, use the X1 shaping tool until the working length determined beforehand is reached.
- Use shaping tool X2 to reconfirm the working length.
- Check the working length again by probing with an ISO 010/015 tool.
- Use the finishing tool X3 until the working length is reached.
- Measure the channel with a manual file.
- If further widening of the apical foramen is necessary, use the appropriate finishing instrument (X4, X5) according to the working length.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal slurry.

### Storage

Store at room temperature, away from moisture and direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

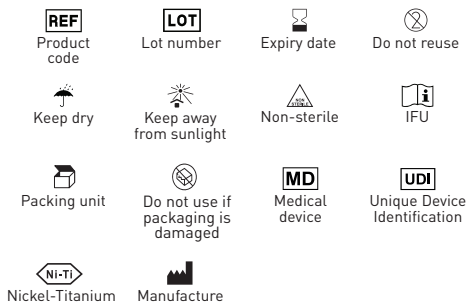
### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.  
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)  
Tel. +39 080 3381004 - Fax. +39 080 3386617  
e-mail: info@dental-world.it / web: www.dental-world.it

Eli-Dent Group S.p.A.  
Via Guido Rossa, 24 A/B/C - 50032  
Borgo San Lorenzo (FI) - www.elidentgroup.it  
Tel. +39 055 600545 - Fax +39 055 601069

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