# IT - Eli-Taper Gold

## Strumenti canalari meccanici

### Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Eli-Dent Group S.p.A.

Via Guido Rossa, 24 A/B/C - 50032

Borgo San Lorenzo (FI) - www.elidentgroup.it

Tel. +39 055 600545 - Fax +39 055 601069

# EN - Eli-Taper Gold

## Mechanical root canal instruments

### Description

Eli-Taper Gold root canal instruments are made with Ni-Ti New-Wire technology, which gives greater elasticity and superior strength compared to normal Ni-Ti instruments. This special alloy This special alloy guarantees extraordinary resistance to cyclic fatigue due to the innovative heat treatment process, as well as greater flexibility, heat treatment process, as well as greater flexibility, ensuring that the canal morphology is maintained.

### Indications

For use in dental practices or hospitals by qualified personnel only.

### Field of application

Shaping and cleaning of root canals.

### Measures

Eli-Taper Gold							
	Ø%		N/cm	RPM	21 mm	25 mm	31 mm
DX	19	3,5-19	3/4	150-300	19 mm	REF: 7300330	
● D1	17	0,2-11	3/4	150-300	7300331	7300431	7300531
○ D2	20	0,4-11,5	1/1,5	150-300	7300332	7300432	7300532
● M1	20	0,7	1/1,5	150-300	7300333	7300433	7300533
● M2	25	0,8	2,5	150-300	7300334	7300434	7300534
● M3	30	0,9	2,5	150-300	7300335	7300435	7300535
● M4	40	0,6	2,5	150-300	7300336	7300436	7300536
● M5	50	0,5	2,5	150-300	7300337	7300437	7300537
DX-M3	--	--	--	150-300	7300338	7300438	7300538

### Packaging

Non-sterile device in blister pack of 6 pieces (assorted or individual sizes)

### Recommended procedure

#### Operating protocol:

- Locate the canal.
- Use the DX opening tool to widen the canal.
- Probe the canal with an ISO 010 or ISO 015 hand file until resistance is encountered resistance.
- Flush the canal using sodium hypochlorite and viscous chelator as a lubricant.
- For shaping, use the D1 instrument until the working length determined previously determined.
- Use shaping tool D2 to reconfirm the working length.
- Check the working length again by probing with an ISO 010/015 tool.
- Use the finishing tool M1 until the working length is reached.
- Measure the channel using a manual file.
- If further widening of the apical foramen is necessary, use the appropriate finishing instrument appropriate finishing instrument (M2, M3) according to the working length.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal slurry.

### Storage

Store at room temperature, away from moisture and

direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

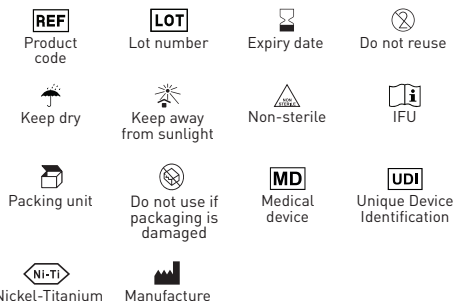
### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.  
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)  
Tel. +39 080 3381004 - Fax. +39 080 3386617  
e-mail: info@dental-world.it / web: www.dental-world.it

 Eli-Dent Group S.p.A.  
Via Guido Rossa, 24 A/B/C - 50032  
Borgo San Lorenzo (FI) - www.elidentgroup.it  
Tel. +39 055 600545 - Fax +39 055 601069

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