IT - ELI C Plus

Strumenti canalari manuali

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Eli-Dent Group S.p.A.

Via Guido Rossa, 24 A/B/C - 50032

Borgo San Lorenzo (FI) - www.elidentgroup.it

Tel. +39 055 600545 - Fax +39 055 601069

EN - ELI C Plus

Manual root canal instruments

Description

ELI C Plus are root canal instruments for use prior to root canal preparation. They facilitate identification of root canal openings, access to the apical third and overcoming calcifications.

Indications

To be used only in a dental practice or hospital environment by qualified personnel.

Field of application

Identification of root canal openings, access to the apical third and overcoming calcifications.

Measures

ELI C Plus			
	21mm	25mm	31mm
○ 06	7300860S	7300870S	7300880S
● 08	7300861S	7300871S	7300881S
● 10	7300862S	7300872S	7300882S
● 12,5	7300863S	7300873S	7300883S
○ 15	7300864S	7300874S	7300884S
15-40	7300865S	7300875S	7300885S

Packaging

Sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Prepare coronal pre-enlargement and access to the canal orifice.
- Explore the canal using small hand files to establish the final working length (in combination with radiographic verification and/or apical detector), confirm patency and verify glide-path.
- Flush thoroughly and frequently with E.D.T.A. and hypochlorite.
- If necessary, expand the glide-path using small hand files.
- Enlarge the apical canal area with small files with frequent and abundant irrigation.
- Proceed with shaping by decreasing the working length according to the desired final taper.

During the endodontic sequence, irrigate repeatedly with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal sludge.

Storage

The blister pack should only be opened at the time of surgery and under controlled aseptic conditions.

Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

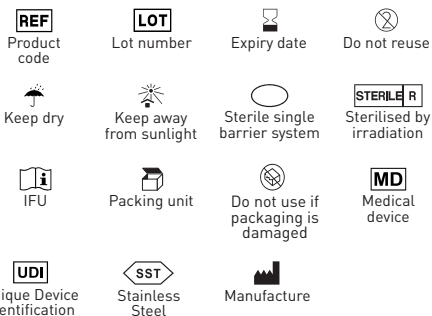
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device you have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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