IT - ELI One Blu

Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Eli-Dent Group S.p.A.

Via Guido Rossa, 24 A/B/C - 50032

Borgo San Lorenzo (FI) - www.elidentgroup.it

Tel. +39 055 600545 - Fax +39 055 601069

EN - ELI One Blu

Mechanical root canal instruments

Description

ELI One Blu instruments are made of Ni-Ti C New Wire (state-of-the-art heat treatment) which gives greater elasticity and superior strength compared to normal instruments.

Indications

To be used only in a dental practice or hospital environment by qualified personnel.

Field of application

Shaping and cleaning root canals.

Measures

ELI One Blu					
	∅	%	21 mm	25 mm	31 mm
● 21	21	0,6	73003405	73003445	73003485
● 25	25	0,8	73003415	73003455	73003495
● 40	40	0,8	73003425	73003465	73003505
21-40	--	--	73003435	73003475	73003515

Packaging

Sterile device in blister pack of 3 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Locate the canal.
- Create access to the canal using an opening tool (ELI Flayre).
- Probe the channel using a manual ISO 010 instrument and expand it to 015, with manual files until resistance is met to determine the working length.
- Start treatment using the 25/08 shaping instrument, irrigate the canal using sodium hypochlorite and viscous chelating agent as lubricant.
- Shape to approx. 2-3 mm, pull out the instrument and repeat the operation in each canal, irrigate again, recapitulate.
- Check the working length using hand instrument 010 with the help of E.D.T.A.
- Bring instrument 25/08 to working length again without forcing. If the tool does not advance use the 21/06 for 2/3 passes until there is no more debris.
- If, however, there is no more debris when using the 25/08 instrument, shape it with the 40/08.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dental slurry.

Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

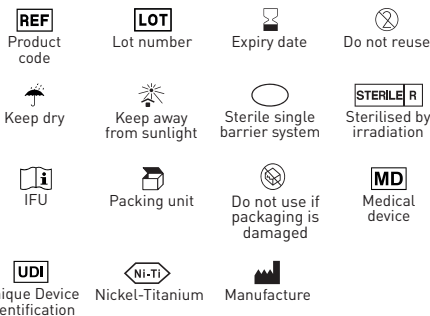
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

 Eli-Dent Group S.p.A.
Via Guido Rossa, 24 A/B/C - 50032
Borgo San Lorenzo (FI) - www.elidentgroup.it
Tel. +39 055 600545 - Fax +39 055 601069

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