# IT - Future Shaper H.A.

## Strumenti canalari meccanici

### Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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# EN - Future Shaper H.A.

## Mechanical root canal instruments

### Description

Future Shaper H.A. root canal instruments are made with Ni-Ti C New-Wire technology, which gives greater elasticity and superior strength compared to normal Ni-Ti instruments. This special alloy This special alloy guarantees extraordinary resistance to cyclic fatigue due to the innovative heat treatment process, as well as greater flexibility. heat treatment process, as well as greater flexibility, ensuring that the canal morphology is maintained.

### Indications

For use in dental practices or hospitals by qualified personnel only.

### Field of application

Shaping and cleaning of root canals.

### Measures

Future Shaper H.A.								
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm	
DX	19	3,5-19	3/4	150-300		19 mm		REF: 166900600
● D1	17	0,2-11	3/4	150-300	166900601	166900610	166900620	
○ D2	20	0,4-11,5	1/1,5	150-300	166900602	166900611	166900621	
● M1	20	0,7	1/1,5	150-300	166900603	166900612	166900622	
● M2	25	0,8	2,5	150-300	166900604	166900613	166900623	
● M3	30	0,9	2,5	150-300	166900605	166900614	166900624	
DX-M3	--	--	--	150-300	166900606	166900615	166900625	
● M4	40	0,6	2,5	150-300	166900607	166900616	166900626	
● M5	50	0,5	2,5	150-300	166900608	166900617	166900627	

### Packaging

Non-sterile device in blister pack of 6 pieces (assorted or individual sizes)

### Recommended procedure

#### Operating protocol:

- Locate the canal.
- Use the DX opening tool to widen the canal.
- Probe the canal with an ISO 010 or ISO 015 hand file until resistance is encountered resistance.
- Flush the canal using sodium hypochlorite and viscous chelator as a lubricant.
- For shaping, use the D1 instrument until the working length determined previously determined.
- Use shaping tool D2 to reconfirm the working length.
- Check the working length again by probing with an ISO 010/015 tool.
- Use the finishing tool M1 until the working length is reached.
- Measure the channel using a manual file.
- If further widening of the apical foramen is necessary, use the appropriate finishing instrument appropriate finishing instrument (M2, M3) according to the working length.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dental slurry.

### Storage

Store at room temperature, away from moisture and direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

Product code

Lot number

Expiry date

Do not reuse

Keep dry

Keep away from sunlight

Non-sterile

IFU

Packing unit

Do not use if packaging is damaged

Medical device

Unique Device Identification

Nickel-Titanium

Manufacture

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for product use careless use.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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