IT - Future Shaper Remove

Strumenti canalari meccanici

Descrizione

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EN - Future Shaper Remove

Mechanical root canal instruments

Description

Future Shaper Remove root canal instruments are instruments made of Ni-Ti indicated for the removal of filling materials such as gutta-percha, cements or other filling material, prior to remodelling the root canal. There are three instruments, each for each coronal third.

Indications

For use only in a dental practice or hospital environment by qualified personnel.

Field of application

Shaping and cleaning of root canals.

Measurements

Future Shaper Remove							
	Ø	%	N/cm	RPM	16 mm	18 mm	22 mm
○ D1	30	9	2,5	150-300	166900120	-	-
○ D2	25	8	2,5	150-300	-	166900131	-
○ D3	20	7	2,5	150-300	-	-	166900142
D1-D3	--	--	--	--	166900123		

Packaging

Non-sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Isolate the operating field with rubber dam.
- Introduce a few drops of chemical softening solution for gutta-percha or other material to be removed into the pulp chamber.
- With the help of the root canal instrument D1 remove the material in the coronal third.
- Use the D2 instrument to remove material in the middle third.
- Use D3 to remove in the apical third.
- Remove the application regularly and rinse alternately with sodium hypochlorite.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials. It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Product code

Lot number

Expiry date

Do not reuse

Keep dry

Keep away from sunlight

Non-sterile

IFU

Packing unit

Do not use if packaging is damaged

Medical device

Unique Device Identification

Nickel-Titanium

Manufacture

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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