IT - Future Broaches

Strumenti canalari manuali

Descrizione

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Misure

| Future Broaches | |
|-----------------|------------|
| Misura | Codice |
| ● xxxxf-0 | 166516600S |
| ○ xxxf-1 | 166516610S |
| ● xxf-2 | 166516620S |
| ● xf-3 | 166516630S |
| ● f-4 | 166516640S |
| ● m-5 | 166516650S |
| ● c-6 | 166516660S |
| Assortito | 166516670S |

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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EN - Future Broaches

Manual root canal instruments

Description

Future Broaches are stainless steel instruments used for extirpation of vital pulp from the root canal.

Indications

For use only in a dental practice or hospital environment by qualified personnel.

Field of application

For extracting the pulp from the root canal of the tooth to be repaired. Recommended for wide, straight canals.

Measures

| Future Broaches | |
|-----------------|------------|
| Size | Code |
| ● xxxxf-0 | 166516600S |
| ○ xxxf-1 | 166516610S |
| ● xxf-2 | 166516620S |
| ● xf-3 | 166516630S |
| ● f-4 | 166516640S |
| ● m-5 | 166516650S |
| ● c-6 | 166516660S |
| Assorted | 166516670S |

Packaging

Sterile device in:

- Blister of 10 pcs.
- Box of 15 blisters

Recommended procedure

Introduce the instrument into the root canal and perform a 180° - 360° rotary movement of the instrument until the pulpal tissues are extirpated and wrapped around the instrument.

During the endodontic sequence, irrigate repeatedly with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal sludge.

Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

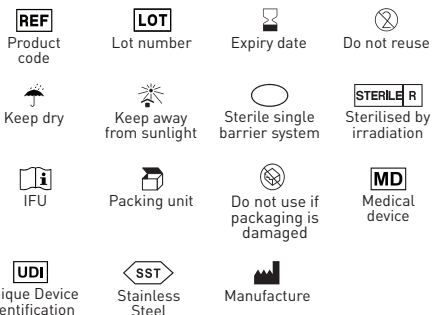
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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