IT - Future Flexo File

Strumenti canalari manuali

Descrizione

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Manual root canal instruments

Description

Future Flexo File are super-flexible steel files indicated for the preparation of irregular canals.

Indications

Only to be used in a dental practice or hospital environment by qualified personnel.

Field of application

For catheterisation and shape definition.

Measures

Future Flexo File		
	21mm	25mm
○ 15	166517182S	166517242S
● 20	166517192S	166517252S
● 25	166517202S	166517262S
● 30	166517212S	166517272S
● 35	166517222S	166517282S
● 40	166517232S	166517292S
15-40	166550712S	166550732S

Packaging

Sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

Use the Future Flexo File for opening and shaping the root canal. Made of stainless steel, they provide greater flexibility for root canal cleaning. The triangular section facilitates the removal of residues. Work in quarter rotation, then withdraw the instrument.

During the endodontic sequence, irrigate repeatedly with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal sludge.

Storage

The blister pack should only be opened at the time of surgery and under controlled aseptic conditions.

Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Product code

Lot number

Expiry date

Do not reuse

Keep dry

Keep away from sunlight

Sterile single barrier system

Sterilised by irradiation

IFU

Packing unit

Do not use if packaging is damaged

Medical device

Unique Device Identification

Stainless Steel

Manufacture

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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