IT - Future Largo & Future Gates

Strumenti canalari meccanici

Descrizione

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EN - Future Largo & Future Gates

Mechanical root canal instruments

Description

Future Largo & Future Gates are stainless steel burs for root canals.

Indications

Only to be used in dental practices or in hospitals by qualified personnel.

Field of application

For enlarging root canals.

Measures

Future Largo		
Ø (mm)	28 mm	32 mm
n°1 - 0,7	166910015S	166910022S
n°2 - 0,9	166910016S	166910023S
n°3 - 1,1	166910017S	166910024S
n°4 - 1,3	166910018S	166910025S
n°5 - 1,5	166910019S	166910026S
n°6 - 1,7	166910020S	166910027S
Ass. n°1 - 6	166910021S	166910028S

Future Gates		
Ø (mm)	28 mm	32 mm
n°1 - 0,5	166910001S	166910008S
n°2 - 0,7	166910002S	166910009S
n°3 - 0,9	166910003S	166910010S
n°4 - 1,5	166910004S	166910011S
n°5 - 1,3	166910005S	166910012S
n°6 - 1,5	166910006S	166910013S
Ass. n°1 - 6	166910007S	166910014S

Packaging

Sterile device in blister pack of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Use in any right angle handpiece to widen the coronal portion of the canal for the massive removal of structure.
- During the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. to remove all necrotic tissue and the organic component of the dental slurry.

Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Product code

Lot number

Expiry date

Do not reuse

Keep dry

Keep away from sunlight

Sterile single barrier system

Sterilised by irradiation

IFU

Packing unit

Do not use if packaging is damaged

Medical device

Unique Device Identification

Stainless Steel

Manufacture

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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