IT - Future Pilot N.H.A. - Future Pilot H.A.

Strumenti canalari meccanici

Descrizione

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EN - Future Pilot N.H.A. - Future Pilot H.A.

Mechanical root canal instruments

Description

Future Pilot H.A. and Future Pilot N.H.A. are Ni-Ti instruments available in Gold, Blue and Coated, with thermal activation. They offer excellent flexibility by easily adapting to the anatomy of most root canals. Instruments with progressively variable taper allow you to work with greater confidence in establishing the glide path.

Indications

To be used only in a dental practice or hospital environment by qualified personnel.

Field of application

Glide path creation.

Measures

Future Pilot NHA							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
○ 016	15	2-6	2,5	300	166900551S	166900561S	166900571S

Future Pilot Heat Activation							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
○ 016	15	2-6	2,5	300	166900751S	166900761S	166900771S

Future Pilot Heat Activation Gold							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
○ 016	15	2-6	2,5	300	166900660S	166900670S	166900680S

Packaging

Sterile device in blister pack of 6 pieces.

Recommended procedure

Operating protocol:

- Locate the canal.
- Use a coronal portion opening tool (Future Flayre).
- Probe with an ISO 010 or 015 manual file.
- Work gently with the Future Pilot H.A. or Future Pilot N.H.A. rotary instrument (ISO 016) in order to obtain a reproducible glide path before canal widening and to reduce the workload of the shaping instruments and risk of breakage.
- Flush the canal using sodium hypochlorite and viscous chelator as lubricant.
- Then follow the operative sequence most suitable for the case to be treated.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. to remove all necrotic tissue and the organic component of the dentinal sludge.

Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients

for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

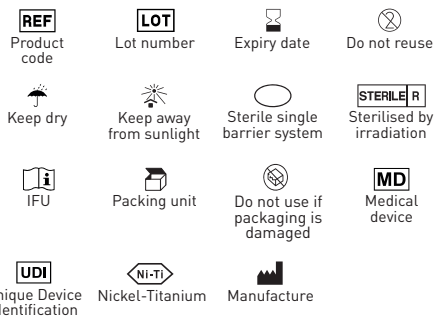
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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