IT - Future Shaper 2 Gold

Strumenti canalari meccanici

Descrizione

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EN - Future Shaper 2 Gold

Mechanical root canal instruments

Description

Future Shaper 2 Gold root canal instruments are made with Ni-Ti Evo Brown technology, which gives greater elasticity and strength and follows even the most pronounced curves harmoniously, compared with normal Ni-Ti instruments. This new technology shortens the time spent changing instruments; in addition, continuous clockwise mechanical instrument movement is recommended.

Indications

For use in dental practices or hospitals by qualified personnel only.

Field of application

Shaping and cleaning of root canals.

Measures

Future Shaper 2 Gold							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
X1	17	0,4	2;5,2	300	1669003005	1669003105	1669003205
X2	25	0,5	2;5,2	300	1669003015	1669003115	1669003215
X3	35	0,7	2;5,2	300	1669003025	1669003125	1669003225
X4	40	0,6	2;5,2	300	1669003035	1669003135	1669003235
X5	50	0,6	2;5,2	300	1669003045	1669003145	1669003245
Assorted	--	--	--	--	1669003055	1669003155	1669003255

Packaging

Sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Locate the canal.
- Use an opening tool to widen the canal or mechanically driven glide-path files (Future Pat).
- Probe the canal with an ISO 010 or ISO 015 hand file until resistance is met.
- Flush the canal using sodium hypochlorite and viscous chelator as lubricant.
- For shaping, use the X1 shaping tool until the working length determined beforehand is reached.
- Use shaping tool X2 to reconfirm the working length.
- Check the working length again by probing with an ISO 010/015 tool.
- Use the finishing tool X3 until the working length is reached.
- Measure the channel with a manual file.
- If further widening of the apical foramen is necessary, use the appropriate finishing instrument (X4, X5) according to the working length.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained there in may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Product code

Lot number

Expiry date

Do not reuse

Keep dry

Keep away from sunlight

Sterile single barrier system

Sterilised by irradiation

IFU

Packing unit

Do not use if packaging is damaged

Medical device

Unique Device Identification

Nickel-Titanium

Manufacture

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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