IT - Future Shaper

Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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EN - Future Shaper

Mechanical root canal instruments

Description

Future Shaper root canal instruments are made with New-Wire Ni-Ti technology and are characterised by high cutting capacity and flexibility.

Indications

Only to be used in dental practices or in hospitals by qualified personnel.

Field of application

Shaping and cleaning of root canals.

Measures

Future Shaper							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
DX	19	3,5-19	3/4	150-300	19 mm	REF: 166900111S	
● D1	17	0,2-11	3/4	150-300	166900112S	166900102S	166900002S
○ D2	20	0,4-11,5	1/1,5	150-300	166900113S	166900103S	166900003S
● M1	20	0,7	1/1,5	150-300	166900114S	166900104S	166900004S
● M2	25	0,8	2,5	150-300	166900115S	166900105S	166900005S
● M3	30	0,9	2,5	150-300	166900116S	166900106S	166900006S
●● M4	40	0,6	2,5	150-300	166900117S	166900107S	166900007S
●● M5	50	0,5	2,5	150-300	166900118S	166900108S	166900008S
DX-M3	--	--	--	150-300	166900110S	166900100S	166900000S

Packaging

Sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Locate the canal.
- Use the DX opening tool to widen the canal.
- Probe the canal with an ISO 010 or ISO 015 hand file until resistance is met and determine the working length.
- Irrigate the canal using sodium hypochlorite and viscous chelator as lubricant.
- For shaping, use the D1 instrument until the working length determined previously is reached.
- Use shaping tool D2 and reconfirm the working length.
- Check the I.L. again by probing with an ISO 010/015 tool.
- Use the finishing tool M1 until the working length is reached.
- If further enlargement of the apical foramen is necessary, use the appropriate finishing instrument (M2, M3, M4 and M5).

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

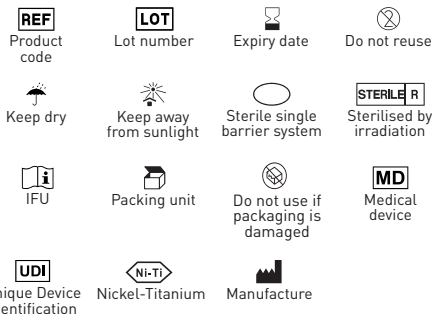
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility

to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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