Fill ENDO Paper

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Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Fill ENDO Paper

Paper points

Description

Paper towel tips are used to dry the root canal before using gutta-percha tips.

Indications for use

- Select the blotting paper tip that best matches the last tool used to shape the canal.
- Dry the canal by gently inserting the paper tips to the final working length.

Precautions

- Product for dental use only.
- Sterile product.
- Non-reusable product.
- Sterility is not guaranteed after the container is opened.
- Keep paper points dry in order to prevent the growth of bacteria make them unusable.
- Carefully follow the recommended storage conditions.
- Do not clean or sterilize the paper points, this may make the device unusable.

Warnings

- Make sure the canal is completely dry after using the paper tips. Improper drying could cause patient sensitivity or delays in procedures.
- Do not apply excessive force during lateral condensation in order to avoid vertical root fracture.
- Ensure proper site preparation by removing infected tissue/bone/dentin in order to prevent infection.
- Follow proper aseptic techniques to prevent contamination.
- Follow proper obturation techniques to avoid procedural accidents that could lead to persistent apical periodontitis infection.
- Be sure to remove the product from the canal after use otherwise post-endodontic periapical inflammation or periapical cysts may occur.
- Do not use distorted or brittle paper tips so as not to compromise canal drying.
- Do not use the product after the expiration date.

You must report any serious incidents occurring in relation to the medical device you supply to the manufacturer and to the competent authority of the Member State in which you are located.


Contraindications

Do not use paper tips on patients who may be sensitive to any of its components.

Storage

- Store at room temperature.
- Store properly in their own packaging.
- Do not disperse in the environment.

Symbols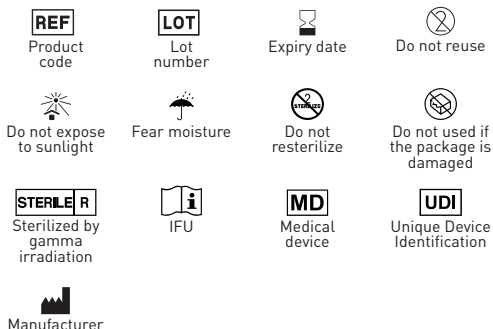

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it