Barbed broach

Strumenti canalari manuali

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Barbed broach

Manual canal instruments

Description

Barbed broach are tools made of stainless steel used for the extraction of the vital pulp from the root canal.

Indications

To be used only in dental clinics or hospitals by qualified personnel.

Application field

For extraction of the pulp from the root canal of the tooth to be repaired. Recommended for wide and straight canals.

Measures

BARBED BROACH	
Size	Code
● xxxxf-0	144516600
○ xxxf-1	144516610
● xxf-2	144516620
● xf-3	144516630
● f-4	144516640
● m-5	144516650
● c-6	144516660
Assorted	144516670

Packaging

- Blister 10 pcs
- Bag 15 blisters

Recommended procedure

Insert the barbed broach into the root canal and rotate the instruments 180° – 360° until pulp tissues are removed so they will remain wrapped around the instrument.

Storage


Store the product at room temperature away from direct light.

Cautions

- Instruments are disposable.
- Repeated use may not be effective and/or cause damage/ inconvenience to the patient.

- Do not use the blister if opened or damaged.

Symbols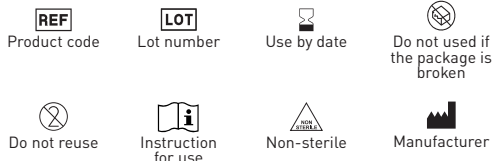

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

 Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

 0476