Pro Flayre

Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Pro Flayre

Mechanical canal instruments

Description

Pro Flayre is a Ni-Ti instrument suitable for the opening of the coronal portion of the canal. This procedure is essential to avoid tensions on instruments and to limit any risks of premature wear.

Indications

To be used only in a dental clinics or in hospitals.

Application field

Shaping and cleaning of root canals.

Measures

PRO FLAYRE					
	∅	%	N/cm	RPM	15 mm
25	25	12	1,5/3	300 - 600	144900420

Packaging

Blister 6 pieces

Recommended procedure

Operating protocol:

- Locate the channel.
- Carry out a sounding with a manual file.
- Use the Pro Flayre tool in order to enlarge the entrance of the root canal thus eliminating the dental protrusions and preserving the homogeneity of the root canal. This step is essential to avoid further stress on the shaping instruments and thus limiting the risk of premature breakage of the instrument. The coronal flare simplifies changes in the working lengths and facilitates the insertion of instruments through the third apical and thus facilitates access to the instrumentation and irrigation of the canal.
- Then follow the more appropriate operating sequence depending on the case dealt with.

Throughout the endodontic sequence, irrigate

repeatedly with hypochlorite (Endo Clor) and E. D. T. A. (Canal Prep) in order to remove all necrotic tissue and organic component dental sludge.


Storage

Store at room temperature, away from direct light sources.

Cautions

- The instruments are recommended for single use.
- If used repeatedly they may not be efficient and/or cause inconvenience.
- Do not use the blister if opened or damaged.

Symbols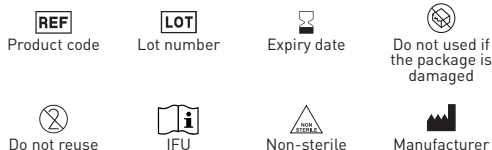

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it