Pro Shaper H.A.

Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

CE 0425

Pro Shaper H.A.

Mechanical canal instruments

Description

The Pro Shaper H. A. canal instruments are made with Ni-Ti C New-Wire technology that gives greater elasticity and superior strength than normal Ni-Ti instruments. This special alloy guarantees an extraordinary resistance to cyclical fatigue thanks to the innovative heat treatment process, as well as greater flexibility, ensuring the maintenance of the canal's morphology.

Indications

To be used only in a dentist studio or in dental clinics hospitals.

Application field

Shaping and cleaning of root canals.

Measures

PRO SHAPER Heat Activation							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
DX	19	3,5-19	2,0-2,5	150-300	19 mm	REF: 144900600	
D1	17	0,2-11	2,0-2,5	150-300	144900601	144900610	144900620
D2	20	0,4-11,5	2,0-2,5	150-300	144900602	144900611	144900621
M1	20	0,7	2,0-2,5	150-300	144900603	144900612	144900622
M2	25	0,8	2,0-2,5	150-300	144900604	144900613	144900623
M3	30	0,9	2,0-2,5	150-300	144900605	144900614	144900624
DX-M3	--	--	--	150-300	144900606	144900615	144900625

Packaging

Blister 6 pcs (assorted or individual sizes).

Recommended procedure

Operative protocol:

- Identify canal.
- Carry out a practicability sounding with a manual file ISO 010 or ISO 015 until resistance is encountered.
- Use the opening instrument DX for the canal enlargement.
- Irrigate the canal using sodium hypochlorite (Endo Clor) and viscous chelator as a lubricant (Canal Prep).
- Use the D1 shaping instrument up to reach the working length previously determined.
- Use the D2 shaping instrument to reconfirm the working length.
- Check the working length again by probing with an ISO 010/015 instrument.
- Use the M1 finishing instrument until obtain the

working length.

- Measure the canal using a manual file.
- If it is necessary a further enlargement of the apical foramen use the appropriate finishing instrument (X4, X5) according to the working length.

Throughout the endodontic sequence, irrigate repeatedly with hypochlorite (Endo Clor) and E. D. T. A. (Canal Prep) in order to remove all necrotic tissue and organic component of smear layer.


Storage

Store at room temperature, away from direct light sources.

Preventive measures

- The instruments are recommended for single use.
- If used repeatedly they may not be efficient and/or cause inconvenience.
- Do not use the blister if opened or damaged.

Symbols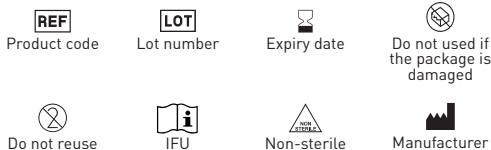

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it