Pro Shaper Remove

Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

REV.1_06-2022

Pro Shaper Remove

Mechanical canal instruments

Description

Pro Shaper Remove canal instrument are tools made of Ni-Ti recommended for the removal of filling materials such as gutta-percha, cements or other filling material, before the re-modelling of the root canal. There are three instruments, each for every third coronal.

Indications

To be used only in dental clinics or in hospitals.

Application field

Shaping and cleaning of root canal.

Measures

PRO SHAPER REMOVE							
	∅	%	N/cm	RPM	16 mm	18 mm	22 mm
○ D1	30	9	2,5	150-300	144900120	-	-
○ D2	25	8	2,5	150-300	-	144900131	-
○ D3	20	7	2,5	150-300	-	-	144900142
D1-D3	--	--	--	--	144900123		

Packaging

6 pcs (assorted or in individual sizes).

Recommended procedure

Operative protocol:

- Isolate the operating field with a dental dam.
- Introduce into the pulp chamber a few drops of solution for chemical softening of gutta-perca or other material to be removed.
- With the help of canal instrument D1 remove the material in the third coronal.
- With the D2 instrument remove the material in the third medium.
- Use D3 for removal in the third apical.
- Regularly remove the application and rinse alternatively with sodium hypochlorite (Endo Clor).

Throughout the endodontic sequence, irrigate repeatedly with hypochlorite (Endo Clor) and E. D. T. A. (Canal Prep) in order to remove all

necrotic tissue and organic component of smear layer.


Storage

Store at room temperature, away from direct light sources.

Preventive measures

- The instruments are recommended for single use.
- If used repeatedly they may not be efficient and/or cause inconvenience.
- Do not use the blister if opened or damaged.

Symbols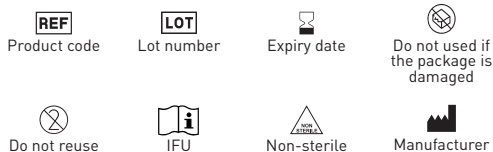

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it