

# Spreader

## Strumenti canalari manuali

### Descrizione

Gli spreader sono strumenti realizzati in acciaio e nickel titanio, dalla forma conica e la punta affilata.

Indicati per la condensazione laterale della guttapercha.

### Indicazioni

Da utilizzare solo in studio odontoiatrico o in ambito ospedaliero da personale qualificato.

### Campo di applicazione

Per la condensazione laterale della guttapercha.

### Misure

SPREADER - Stainless Steel		
	21mm	25mm
○ 15	144513142	144513242
● 20	144513112	144513212
● 25	144513152	144513252
● 30	144513122	144513222
● 35	144513162	144513262
● 30	144513132	144513232
15-40	144550832	144550842

SPREADER - Nickel-Titanium	
	25mm
○ 15	144443242
● 20	144443212
● 25	144443252
● 30	144443222
● 35	144443262
● 30	144443232
15-40	144450842

### Confezionamento

Blister da 6 pezzi (assortiti o singole misure)

### Procedura raccomandata

*Protocollo operativo:*

Inserire un cono master di misura e conicità appropriate fino a che non si avverte una resistenza all'inserimento. Il cono master viene scelto in relazione allo strumento impiegato per sagomare il canale.

Verificare radiograficamente il posizionamento corretto ed estrarre il cono. Miscelare il cemento canalare ed inserirlo nel canale, quindi introdurre nuovamente il cono master nel canale. Affiancare al cono master coni accessori che vengono compattati lateralmente mediante lo spreader.


### Conservazione

Conservare a temperatura ambiente, lontano da fonti di luce diretta.

### Precauzioni

- Gli strumenti sono consigliati monouso.
- Se utilizzati ripetutamente potrebbero non essere efficienti e/o causare danni/inconvenienti al paziente.
- Non utilizzare il blister se aperto o danneggiato.

### Simboli

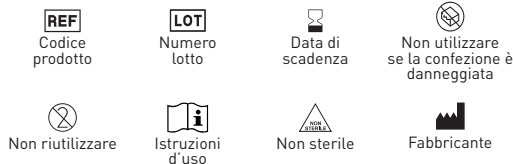

### Garanzia

Dental World sostituirà il prodotto che risulti essere difettoso.

Dental World non accetta la responsabilità per alcun danno o perdita, diretto o conseguente scaturito da uso incauto o l'inabilità ad usare il prodotto descritto. Prima di utilizzare, è responsabilità dell'utente determinare l'appropriatezza del prodotto per il suo uso. L'utente si assume la responsabilità per uso incauto.

 Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

# Spreader

## Manual root canal instruments

### Description

Spreaders are instruments made of steel and nickel titanium, with a conical shape and sharp tip. Suitable for lateral condensation of gutta-percha.

### Indications

For use in dental practices or hospitals by qualified personnel only.

### Field of application

For lateral condensation of gutta-percha.

### Measures

SPREADER - Stainless Steel		
	21mm	25mm
○ 15	144513142	144513242
● 20	144513112	144513212
● 25	144513152	144513252
● 30	144513122	144513222
● 35	144513162	144513262
● 30	144513132	144513232
15-40	144550832	144550842

SPREADER - Nickel-Titanium	
	25mm
○ 15	144443242
● 20	144443212
● 25	144443252
● 30	144443222
● 35	144443262
● 30	144443232
15-40	144450842

### Packaging

Blister 6 pcs (assorted or individual sizes).

### Recommended procedure

#### Operative protocol:

Insert a master cone of appropriate size and taper until resistance to insertion is felt. The master taper is chosen in relation to the instrument used to shape the canal.

Radiographically verify proper placement and extract the cone. Mix the canal cement and insert it into the canal, then introduce the master cone back into the canal. Flank the master cone with accessory cones that are laterally compacted using the spreader.

### Storage


Store at room temperature, away from direct light sources.

### Preventive measures

- The instruments are recommended for single use.

- If used repeatedly they may not be efficient and/or cause inconvenience.
- Do not use the blister if opened or damaged.

### Symbols

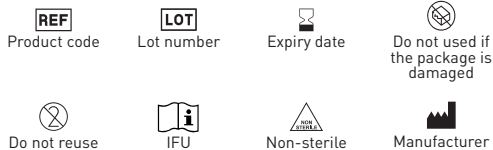

### Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: [info@dental-world.it](mailto:info@dental-world.it) / web: [www.dental-world.it](http://www.dental-world.it)

