# IT - Flexo File

## Strumenti canalari manuali

### Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

# EN - Flexo File

## Manual root canal instruments

### Description

Flexo File are super-flexible steel files indicated for the preparation of irregular canals.

### Indications

Only to be used in a dental practice or hospital environment by qualified personnel.

### Field of application

For catheterisation and shape definition.

### Measures

FLEXO FILE		
	25mm	25mm
○ 15	144517182	144517242
● 20	144517192	144517252
● 25	144517202	144517262
● 30	144517212	144517272
● 35	144517222	144517282
● 40	144517232	144517292
15-40	144550712	144550732

### Packaging

Non-sterile device in blister packs of 6 pieces (assorted or individual sizes).

### Recommended procedure

#### Operating protocol:

Use the Flexo File for opening and shaping the root canal. Made of stainless steel, they provide greater flexibility for root canal cleaning. The triangular section facilitates the removal of residues. Work in quarter rotation, then withdraw the instrument.

During the endodontic sequence, irrigate repeatedly with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dentinal sludge.

### Storage

Store at room temperature, away from moisture and direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination.

Failure to do so may result in cross-infection.

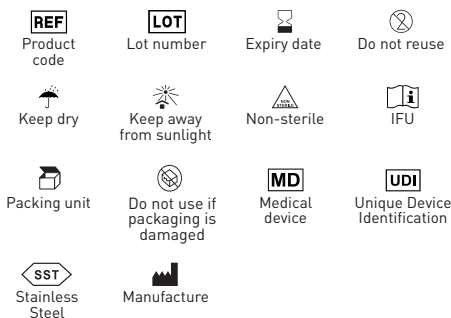
### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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