IT - Pro C Plus

Strumenti canalari manuali

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Pro C Plus

Manual root canal instruments

Description

Pro C Plus are root canal instruments for use prior to root canal preparation. They facilitate identification of root canal openings, access to the apical third and overcoming calcifications.

Indications

To be used only in a dental practice or hospital environment by qualified personnel.

Field of application

Identification of root canal openings, access to the apical third and overcoming calcifications.

Measures

PRO C PLUS			
	21mm	25mm	31mm
○ 06	144517183	144517243	144517245
● 08	144517193	144517253	144517255
● 10	144517203	144517263	144517265
● 12,5	144517213	144517273	144517275
○ 15	144517223	144517283	144517285
15-40	144550713	144550733	144550735

Packaging

Non-sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Prepare coronal pre-enlargement and access to the canal orifice.
- Explore the canal using small hand files to establish the final working length (in combination with radiographic verification and/or apical detector), confirm patency and verify glide-path.
- Flush thoroughly and frequently with E.D.T.A. (Canal Prep) and hypochlorite (Endo Clor).
- If necessary, expand the glide-path using small hand files.
- Enlarge the apical canal area with small files with frequent and abundant irrigation.
- Proceed with shaping by decreasing the working length according to the desired final taper.

During the endodontic sequence, irrigate repeatedly with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dental sludge.

Storage

Store at room temperature, away from moisture and

direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

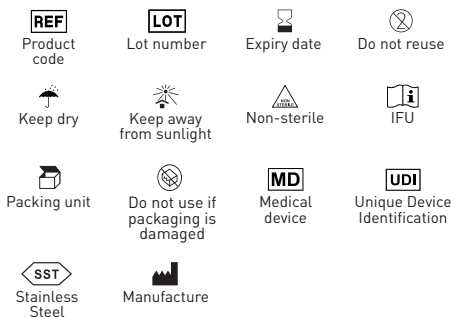
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device you have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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