# IT - Eli-Proc Blu

## Strumenti canalari meccanici

### Descrizione

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# EN - Eli-Proc Blu

## Mechanical root canal instruments

### Description

Eli-Proc Blu root canal instruments are made with Ni-Ti C New-Wire technology, which provides greater elasticity and superior strength compared to normal Ni-Ti instruments.

### Indications

For use in dental practices or hospitals by qualified personnel only.

### Field of application

Shaping and cleaning root canals.

### Measures

Eli-Proc Blu					
	∅	%	21 mm	25 mm	31 mm
● R25	25	0,8	<b>7300100</b>	<b>7300200</b>	<b>7300300</b>
● R40	40	0,6	<b>7300101</b>	<b>7300201</b>	<b>7300301</b>
● R50	50	0,5	<b>7300102</b>	<b>7300202</b>	<b>7300302</b>
R25-R50	--	--	<b>7300103</b>	<b>7300203</b>	<b>7300303</b>

### Packaging

Non-sterile device in blister pack of 6 pieces (assorted or individual sizes).

### Recommended procedure

#### Operating protocol:

- Prepare a root canal access.
- Select the Eli-Proc Blu instrument to be used for canal preparation.
- If the canal is considered narrow use the R25 instrument. If the canal is considered medium use the R40 instrument. If the canal is considered wide use the R50 instrument.
- Preparation with the R25 instrument: estimate the length of the root canal with the help of the radiograph and adjust the stop on the instrument to approx. 2/3 of the estimated length.
- Preparation with the R40-R50 instrument: electronically determine the working length and adjust the stop on the instrument to this length.
- Irrigate the canal with E.D.T.A. (Canal Prep) and hypochlorite (Endo Clor).
- Insert the instrument into the canal and move it slowly apically, applying very light pressure; the instrument can then move forward easily. The amplitude of the up and down movements (passages) must not exceed 3 mm.
- After 3 passes clean the instrument, irrigate the canal and check its patency.
- Repeat until 2/3 of the estimated working length is reached.
- If using the R25 instrument, determine the working length electronically. If you are using the R40 and R50, after preparing 2/3 of the channel length, electronically check the working length initially established.
- Subsequently, recapitulate until the entire working length is reached.
- Determine the apical diameter by inserting a hand instrument ISO higher than the Eli-Proc Blu instrument inserted up to the working length. If the hand instrument is inserted up to 0.7 mm before the working length, no further extension of the apical area is necessary. If the hand instrument can be inserted up to the working length, it is usually necessary to finish the preparation with a larger Eli-Proc Blu instrument.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to

remove all necrotic tissue and the organic component of the dentinal slurry.

### Storage

Store at room temperature, away from moisture and direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

Product code

Lot number

Expiry date

Do not use

Keep dry

Keep away from sunlight

Non-sterile

IFU

Packing unit

Do not use if packaging is damaged

Medical device

Unique Device Identification

Nickel-Titanium

Manufacture

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility

to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

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**ELI-DENT** GROUP S.p.A.
**Dental World S.r.l.**