IT - Eli-Taper Gold

Strumenti canalari meccanici

Descrizione

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EN - Eli-Taper Gold

Mechanical root canal instruments

Description

Eli-Taper Gold root canal instruments are made with New-Wire Ni-Ti technology and are characterised by high cutting capacity and flexibility.

Indications

Only to be used in dental practices or in hospitals by qualified personnel.

Field of application

Shaping and cleaning of root canals.

Measures

| Eli-Taper Gold | | | | | | | |
|----------------|----|----------|-------|---------|---------|--------------|---------|
| | Ø | % | N/cm | RPM | 21 mm | 25 mm | 31 mm |
| DX | 19 | 3,5-19 | 3/4 | 150-300 | 19 mm | REF: 7300330 | |
| ● D1 | 17 | 0,2-11 | 3/4 | 150-300 | 7300331 | 7300431 | 7300531 |
| ○ D2 | 20 | 0,4-11,5 | 1/1,5 | 150-300 | 7300332 | 7300432 | 7300532 |
| ● M1 | 20 | 0,7 | 1/1,5 | 150-300 | 7300333 | 7300433 | 7300533 |
| ● M2 | 25 | 0,8 | 2,5 | 150-300 | 7300334 | 7300434 | 7300534 |
| ● M3 | 30 | 0,9 | 2,5 | 150-300 | 7300335 | 7300435 | 7300535 |
| ●● M4 | 40 | 0,6 | 2,5 | 150-300 | 7300336 | 7300436 | 7300536 |
| ●● M5 | 50 | 0,5 | 2,5 | 150-300 | 7300337 | 7300437 | 7300537 |
| DX-M3 | -- | -- | -- | 150-300 | 7300338 | 7300438 | 7300538 |

Packaging

Non-sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Locate the canal.
- Use the DX opening tool to widen the canal.
- Probe the canal with an ISO 010 or ISO 015 hand file until resistance is met and determine the working length.
- Irrigate the canal using sodium hypochlorite (Endo Clor) and viscous chelator as lubricant (Canal Prep).
- For shaping, use the D1 instrument until the working length determined previously is reached.
- Use shaping tool D2 and reconfirm the working length.
- Check the I.L. again by probing with an ISO 010/015 tool.
- Use the finishing tool M1 until the working length is reached.
- If further enlargement of the apical foramen is necessary, use the appropriate finishing instrument (M2, M3, M4 and M5).

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

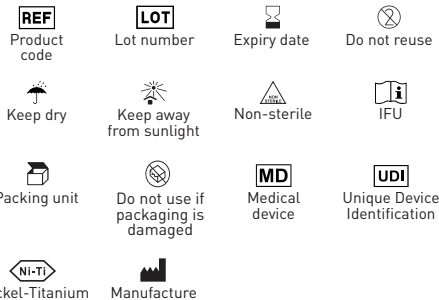
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

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ELI-DENT GROUP S.p.A.
Dental World S.r.l.