IT - Pro Dual S

Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Pro Dual S

Mechanical root canal instruments

Description

Pro Dual S root canal instruments are heat-activated Ni-Ti Blue files that include an opening instrument and a dedicated glide path instrument. The special feature of the shaping/finishing files is a single blade (3-17mm) that improves cutting efficiency and debris removal and a non-cutting tip (0-3mm) that prevents perforation and thus allows for extremely safe working conditions.

Indications

For use only in a dental practice or hospital environment by qualified personnel.

Field of application

Opening, glide-path, shaping and cleaning root canals.

Measures

Pro Dual S							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
DX	19	3,5-19	2,0-2,5	150-350	19 mm	REF: 144900750	
● 020/04	20	4	2,0-2,5	150-350	144900752	144900762	144900772
● 025/04	25	4	2,0-2,5	150-350	144900753	144900763	144900773
● 025/06	25	6	2,0-2,5	150-350	144900756	144900764	144900774
ASS	--	--	--	150-350	144900755	144900765	144900775

Packaging

Non-sterile device in blister packs of 5 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Locate the canal.
- Create coronal and root access using the DX opening instrument.
- In the presence of a viscous chelator as a lubricant (Canal Prep), make a probing path with a hand file 010 and expand it to 016 with the Pro Pilot H.A. or Pro Pilot N.H.A. instrument in order to obtain a reproducible glide path.
- Irrigate with sodium hypochlorite (Endo Clor) and continue the endodontic sequence with shaping/finishing until the working length is reached using the 20/05, 25/04, and 25/06 instruments depending on the case to be treated, preferring the 04 taper for narrow, very curved and apically curved canals and the 06 taper for most clinical cases.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials. It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-

infection.

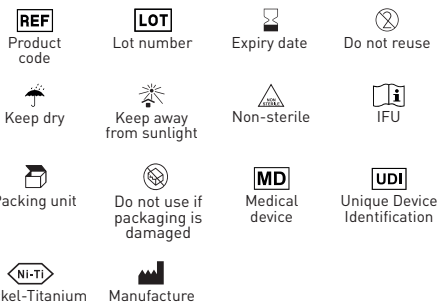
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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