# IT - Pro Flex Evo H.A.

## Strumenti canalari meccanici

### Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

# EN - Pro Flex Evo H.A.

## Mechanical root canal instruments

### Description

Pro Flex Evo H.A. root canal instruments are designed for practical and fast root canal treatment. The Nickel Titanium Heat Brown alloy ensures strong and elastic instruments that harmoniously follow even the most pronounced curves without neglecting their penetrative power.

### Indications

Only to be used in dental practices or in hospitals by qualified personnel.

### Field of application

Shaping and cleaning of root canals.

### Measures

PRO FLEX EVO Heat Activation								
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm	
ER	--	--	2-2,5	350	19 mm	REF: 144900372		
● E1	20	4	2-2,5	150-350	144900373	144900383	144900393	
● E2	25	4	2-2,5	150-350	144900374	144900384	144900394	
● E3	30	4	2-2,5	150-350	144900375	144900385	144900395	
● E4	35	4	2-2,5	150-350	144900376	144900386	144900396	
●● E5	25	6	2-2,5	150-350	144900377	144900387	144900397	
E1-E5	--	--	2-2,5	150-350	144900378	144900388	144900398	

### Packaging

Non-sterile device in blister packs of 6 pieces (assorted or individual sizes).

### Recommended procedure

#### Operating protocol:

- In the first apical millimetres using the ER instrument, create access to the canal orifice and irrigate.
- Gently perform a passive probing of the coronal 2/3 with a hand instrument ISO 010 or 015 (mechanical files such as Pro Pat can be used alternatively).
- Use sodium hypochlorite (Endo Clor) and viscous chelator as a lubricant (Canal Prep) to irrigate the canal.
- Work carefully and gently until a reproducible glide-path is obtained.
- For small canals, shape with E1-E2 until the working length is reached and irrigate.
- Use E3 for medium-sized canals, reconfirm the working length, recap and irrigate again.
- For large canals shape with the E4-E5 instruments.

During the entire endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the mud dentinal sludge.

### Storage

Store at room temperature, away from moisture

and direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials. It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

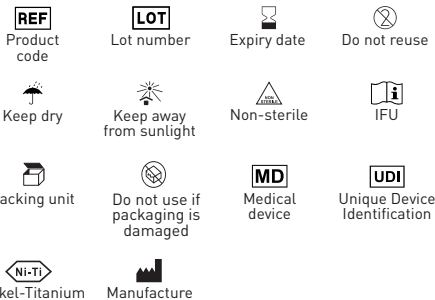
### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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