# IT - Pro Flex N.H.A.

## Strumenti canalari meccanici

### Descrizione

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# EN - Pro Flex N.H.A.

## Mechanical root canal instruments

### Description

Pro Flex N.H.A. root canal instruments are thermally activated Ni-Ti and Coated files designed for complete root canal preparation. The kit includes an opening instrument, the Pro Pilot N.H.A. glide-path instrument and four rotary files for shaping and finishing all types of canals.

### Indications

For use only in a dental practice or hospital environment by qualified personnel.

### Field of application

Glide path, shaping and cleaning root canals.

### Measures

Pro Flex NHA							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
R	20	10	2,5	350	17 mm	REF: 144900550	
○ 016	15	2-6	2,5	300	144900551	144900561	144900571
● D1	20	4	2,5	150-350	144900552	144900562	144900572
● D2	25	4	2,5	150-350	144900553	144900563	144900573
● D3	30	4	2,5	150-350	144900554	144900564	144900574
● D4	25	6	2,5	150-350	144900555	144900565	144900575
R-D4	--	--	2,5	--	144900556	144900566	144900576
● D5	35	4	2,5	150-350	144900557	144900567	144900577
● D6	40	4	2,5	150-350	144900558	144900568	144900578
● D7	50	4	2,5	150-350	144900559	144900569	144900579
D5-D7	--	--	2,5	--	144900560	144900570	144900580

### Packaging

Non-sterile device in blister packs of 6 pieces (assorted or individual sizes).

### Recommended procedure

#### Operating protocol:

- Locate the canal.
- Gently perform a passive probing of the coronal 2/3 with a hand instrument ISO 010 or 015 (mechanical files such as Pro Pat can be used alternatively).
- Using the R instrument, create access to the canal orifice and irrigate.
- Use sodium hypochlorite (Endo Clor) and viscous chelator as a lubricant (Canal Prep) to irrigate the canal.
- To achieve a reproducible glide-path before enlargement and to reduce the workload of the shaping instruments and the risk of breakage, work gently with the Pro Pilot N.H.A. rotary instrument (ISO 016).
- Shape with D1 until working length is reached, irrigate.
- Use D2 to reconfirm the working length, recapitulate and irrigate again, especially in more curved canals.
- For large canals use D3/D4 without oscillating movement.
- For finishing rather large root canals use instruments D5, D6 and D7.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A.

(Canal Prep) in order to remove all necrotic tissue and the organic component of the dentinal slurry.

### Storage

Store at room temperature, away from moisture and direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

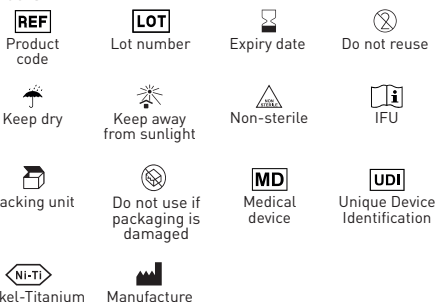
### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility

to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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