IT - Pro One 2 H.A.

Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Pro One 2 H.A.

Mechanical root canal instruments

Description

Pro One 2 H.A. instruments, made of thermally activated Ni-Ti Gold and designed to reduce shaping time, offer the possibility of use as a single root canal instrument.

Indications

To be used only in a dental practice or hospital environment by qualified personnel.

Field of application

Shaping and cleaning root canals.

Measures

PRO ONE 2 Heat Activation					
	∅	%	21 mm	25 mm	31 mm
● 20	25	7	144900630	144900640	144900650
● 25	20	7	144900631	144900641	144900651
● 35	25	6	144900632	144900642	144900652
○ 45	30	5	144900633	144900643	144900653
20-45	--	--	144900643	144900644	144900654

Packaging

Non-sterile device in blister pack of 4 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Create a straight coronal and root access.
- In the presence of a viscous chelator as a lubricant (Canal Prep), probe the canal with a manual ISO 010 file and expand it to 015 with a manual or mechanical instrument (Pro Pilot H.A. or Pro Pilot N.H.A.) in order to obtain a reproducible glide path.
- Identify the Pro One 2 H.A. instrument to be used according to the size of the canal to be treated.
- Start shaping for 2-3 mm in the presence of sodium hypochlorite (Endo Clor).
- Remove the instrument and repeat the operation in the canal, irrigate.
- Continue for 2/3 passes to widen the canal to two thirds of its length.
- Remove any coronal interference.
- Bring the file to full working length in one or more passes. When the length is reached, remove the file to avoid excessive enlargement of the hole.
- Inspect the apical grooves, proceed with shaping and ensure the canal is free of debris.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

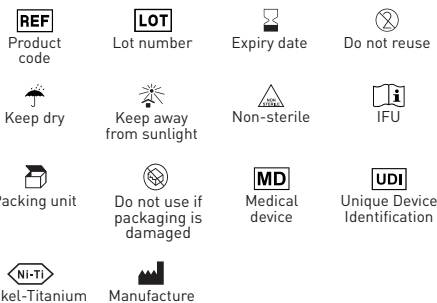
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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