# IT - Pro Shaper 2 H.A.

## Strumenti canalari meccanici

### Descrizione

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# EN - Pro Shaper 2 H.A.

## Mechanical root canal instruments

### Description

The Pro Shaper 2 H.A. root canal instruments are made of thermally activated Ni-Ti BLU and are able to treat a wide range of root canals, even narrow and particularly curved ones, allowing, thanks to their cutting efficiency, a considerable reduction in time spent on shaping.

### Indications

Only to be used in a dental practice or hospital environment by qualified personnel.

### Field of application

Shaping and cleaning of root canals.

### Measures

PRO SHAPER 2 Heat Activation							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
● G1	17	4	2,5	150-350	144900661	144900671	144900681
● G2	25	6	2,5	150-350	144900662	144900672	144900682
● G3	30	7	2,5	150-350	144900663	144900673	144900683
● G4	40	6	2,5	150-350	144900664	144900674	144900684
● G5	30	6	2,5	150-350	144900665	144900675	144900685
G1-G5	--	--	--	150-350	144900666	144900676	144900686

### Packaging

Non-sterile device in blister packs of 6 pieces (assorted or individual sizes).

### Recommended procedure

#### Operating protocol:

- Locate the canal.
- Use an opening tool for channel widening or mechanically operated glide-path files (Pro Pilot H.A. or Pro Pilot N.H.A.).
- Probe the canal with an ISO 010 or ISO 015 hand file until resistance is met.
- Flush the canal using sodium hypochlorite (Endo Clor) and viscous chelator as lubricant (Canal Prep).
- For shaping, use the G1 shaping tool until the working length determined beforehand is reached.
- Use shaping tool G2 to reconfirm the working length.
- Check the working length again by probing with an ISO 010/015 tool.
- Use the finishing tool G3 until the working length is reached.
- Measure the channel with a manual file.
- If further widening of the apical foramen is necessary, use the appropriate finishing instrument (G4, G5) according to the working length.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dentinal slurry.

### Storage

Store at room temperature, away from moisture and direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials. It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

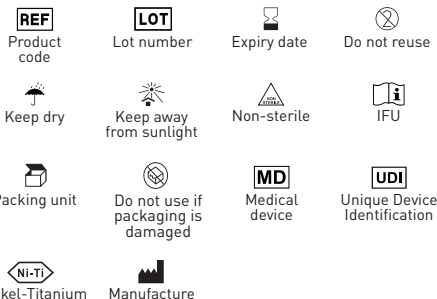
### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.  
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)  
Tel. +39 080 3381004 - Fax. +39 080 3386617  
e-mail: info@dental-world.it / web: www.dental-world.it

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