# IT - Eli-True

## Strumenti canalari meccanici

### Descrizione

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# EN - Eli-True

## Mechanical root canal instruments

### Description

Eli-True root canal instruments are suitable for treating and finishing most root canals and are particularly suitable in the simultaneous technique, without the need for any early enlargement of the coronal third. It consists of 4 instruments used in progression that allow the canal to be prepared up to the apex without substantially changing the morphology of the canal itself.

### Indications

For use only in a dental office or hospital setting by qualified personnel.

### Field of application.

Shaping and cleaning of root canals.

### Measurements

Eli-True							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
● 10/04	10	04	1,2	280	7300310S	7300410S	7300510S
○ 15/05	15	05	1,3	280	7300311S	7300411S	7300511S
● 20/06	20	06	2,1	250-350	7300312S	7300412S	7300512S
● 25/06	25	06	2,3	250-350	7300313S	7300413S	7300513S
● 30/05	30	05	1,2	250-350	7300314S	7300414S	7300514S
● 30/06	30	06	1,2	250-350	7300321S	7300421S	7300521S
● 35/04	35	04	1,2	250-350	7300315S	7300415S	7300515S
● 35/06	35	06	1,2	250-350	7300322S	7300422S	7300522S
● 40/04	40	04	1,6	250-350	7300316S	7300416S	7300516S
● 40/06	40	06	1,5	250-350	7300323S	7300423S	7300523S
10/20	--	--	--	250-350	7300324S	7300424S	7300524S

### Packaging

Sterile device in blister packs of 6 pieces (assorted or single sizes).

### Recommended procedure.

#### Operative protocol:

- After exploring the canal up to the apex with an ISO 010 manual file, continue with the following sequence of instruments, which are used from the smallest to the largest: 10/04, 15/05, 20/06, 25/06. This sequence allows a gradual enlargement of the entire canal in an apico-coronal manner and is valid for all canal types. It allows already with the first instrument, the creation of an immediate truncation up to the apex.
- If you feel a slowdown in the progression of the instrument, retract it 1-2 mm to make it first work passively in "brushing" action, selectively remove interference and let it advance further toward the foramen. If necessary, that is, if obstruction is again felt, the movement will be repeated.
- In the presence of canals where it was possible to easily reach the apex with an ISO 015 or 020 manual file, you may begin with the instrument having the tip diameter corresponding to that of the manual file that reached the apex.
- If it is determined that the apex measures more than 025, the basic sequence of the first four instruments can be joined by instruments of gauge 030, 035 or 040 with a taper of .06, which will ensure adequate preparations for particularly wide canals in a logic of progressive flaring proportional to the apical opening.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dental slurry.

### Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

Product code

Lot number

Expiry date

Do not reuse

Keep dry

Keep away from sunlight

Sterile single barrier system

Sterilised by irradiation

IFU

Packing unit

Do not use if packaging is damaged

Medical device

Unique Device Identification

Nickel-Titanium

Manufacture

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

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**ELI-DENT GROUP S.p.A.**
Dental World S.r.l.