# IT - Pro Flex H.A.

## Strumenti canalari meccanici

### Descrizione

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# EN - Pro Flex H.A.

## Mechanical root canal instruments

### Description

Pro Flex H.A. root canal instruments comprise just three instruments that can be used in succession to shape and finish all types of canals.

### Indications

For use only in a dental practice or hospital environment by qualified personnel.

### Field of application

Shaping and cleaning of root canals.

### Measures

PRO FLEX Heat Activation							
	∅	%	N/cm	RPM	21 mm	25 mm	31 mm
R	--	--	2,5	350	17 mm	REF: 1449003995	
● S1	20	0,4	2,5	250-300	1449004005	1449004105	1449004155
● S2	25	0,6	2,5	250-300	1449004015	1449004115	1449004165
● S3	35	0,4	2,5	250-300	1449004025	1449004125	1449004175
10-S3	--	--	2,5	--	1449004035	1449004135	1449004185

### Packaging

Sterile device in blister packs of 6 pieces (assorted or individual sizes).

### Recommended procedure

#### Operating protocol:

- Locate the canal.
- Using the R instrument, create access to the canal orifice and irrigate.
- Gently perform a passive probing of the coronal 2/3 with a hand instrument ISO 010 or 015 (mechanical files such as Pro Pat can be used alternatively).
- Use sodium hypochlorite (Endo Clor) and viscous chelator as a lubricant (Canal Prep) to irrigate the canal.
- Work carefully and gently until a reproducible glide-path is obtained.
- Shape with S1 until working length is reached, irrigate.
- Use S2 to reconfirm the working length, recapitulate and irrigate again, especially in more curved canals.
- For large canals use S3 without oscillating movement.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dentinal slurry.

### Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

### Precautions

Before use, it is recommended to check with

patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

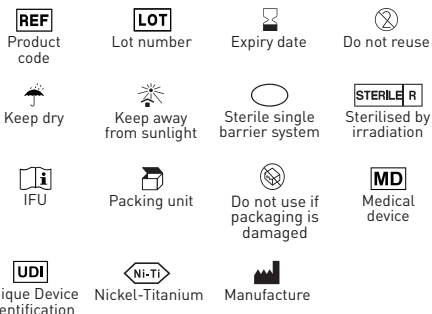
### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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