IT - Pro Flex N.H.A.

Strumenti canalari meccanici

Descrizione

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EN - Pro Flex N.H.A.

Mechanical root canal instruments

Description

Pro Flex N.H.A. root canal instruments are thermally activated Ni-Ti and Coated files designed for complete root canal preparation. The kit includes an opening instrument, the Pro Pilot N.H.A. glide-path instrument and four rotary files for shaping and finishing all types of canals.

Indications

For use only in a dental practice or hospital environment by qualified personnel.

Field of application

Glide path, shaping and cleaning root canals.

Measures

Pro Flex NHA								
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm	
R	20	10	2,5	350	17 mm	REF: 1449005505		
○ 016	15	2-6	2,5	300	1449005515	1449005615	1449005715	
● D1	20	4	2,5	150-350	1449005525	1449005625	1449005725	
● D2	25	4	2,5	150-350	1449005535	1449005635	1449005735	
● D3	30	4	2,5	150-350	1449005545	1449005645	1449005745	
● D4	25	6	2,5	150-350	1449005555	1449005655	1449005755	
R-D4	--	--	2,5	--	1449005565	1449005665	1449005765	
● D5	35	4	2,5	150-350	1449005575	1449005675	1449005775	
● D6	40	4	2,5	150-350	1449005585	1449005685	1449005785	
● D7	50	4	2,5	150-350	1449005595	1449005695	1449005795	
D5-D7	--	--	2,5	--	1449005605	1449005705	1449005805	

Packaging

Sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Opening protocol:

- Locate the canal.
- Gently perform a passive probing of the coronal 2/3 with a hand instrument ISO 010 or 015 (mechanical files such as Pro Pat can be used alternatively).
- Using the R instrument, create access to the canal orifice and irrigate.
- Use sodium hypochlorite (Endo Clor) and viscous chelator as a lubricant (Canal Prep) to irrigate the canal.
- To achieve a reproducible glide-path before enlargement and to reduce the workload of the shaping instruments and the risk of breakage, work gently with the Pro Pilot N.H.A. rotary instrument (ISO 016).
- Shape with D1 until working length is reached, irrigate.
- Use D2 to reconfirm the working length, recapitulate and irrigate again, especially in more curved canals.
- For large canals use D3/D4 without oscillating movement.
- For finishing rather large root canals use instruments D5, D6 and D7.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A.

(Canal Prep) in order to remove all necrotic tissue and the

organic component of the dentinal slurry.

Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

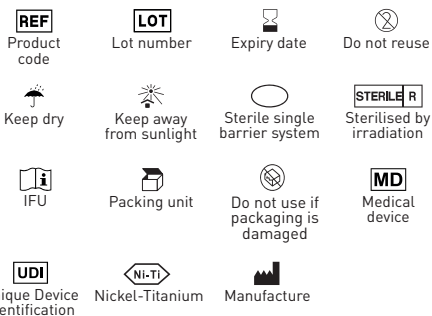
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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