IT - Pro Pat EVO

Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Pro Pat EVO

Mechanical root canal instruments

Description

Pro Pat EVO root canal instruments are designed for greater flexibility and resistance to cyclical fatigue. The special square section design ensures that the file thickness is taken care of and allows for high strength. The 03 taper reduces resistance in the downward direction. These new rotary instruments guarantee excellent cutting performance and debris removal.

Indications

For use in dental practices or hospitals by qualified personnel only.

Field of application

Shaping and cleaning root canals.

Measures

Pro Pat EVO						
	∅	%	N/cm	RPM	21 mm	25 mm
● 010	10	3	3	250	144900464S	144900474S
○ 015	15	3	3	250	144900465S	144900475S

Packaging

Sterile device in blister pack of 6 pieces (individual sizes).

Recommended procedure

Operating protocol:

- In the absence of coronal interference, probe with an ISO 010 steel K-File until it moves freely in the canal (glidepath) and thus verify the working length.
- After establishing the patency of the apical foramen and confirming the working length, use the 3 Pro Pat files to the same length, thus pre-flaring in a few seconds.
- If the working length can be taken with a K-File ISO 015, go directly to the Pro Pat EVO 0.16 mm.
- If the working length can be taken with a K-File ISO 020, we switch directly to Pro Pat EVO 0.19 mm.
- After obtaining a 19-gauge foramen, we can safely take the rotary system we are accustomed to using, such as Pro Shaper or others, to the same working length and safely perform the correct shaping.
- After the use of each instrument, it is advisable to perform a thorough irrigation with sodium hypochlorite (Endo Clor) and viscous chelator as a lubricant (Canal Prep).
- If the canal cannot be probed all the way to the foramen from the beginning, due to the presence of coronal interference or very accentuated curves in the apical third, the Pro Pat EVOs are used in step back, carrying them as far as the canal accommodates them without engaging the tip, and then probing with the pre-curved K-File 10 file, taking the working length and proceeding in the manner described above.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A.

(Canal Prep) in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room

temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

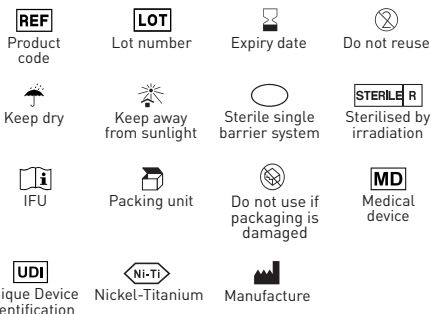
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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